

EDTA

17% EDTA SOLUTION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

17% EDTA Solution buffered to neutral pH used for preparation of roots canals by rapid cleansing and enlarging of roots canals.

Composition :

Disodium Edetate, purified water.

Indications :

For root canal cleansing and shaping during instrumentation. EDTA Solution helps in opening calcified canals.

Instructions for Use :

Transfer portion of solution from the bottle to a disposable syringe fitted with a disposable needle. Insert the needle into the root canal until resistance is felt and withdraw about 2mm. Inject the solution slowly into the canal to irrigate debris and to chelate the canal walls.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Contraindications : It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions : Avoid injecting solution beyond apex of canal.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref: 40004

1 x 15ml Glass Bottle, Dispensing Dropper.

Ref: 40004-1

1 x 100ml Glass Bottle, Dispensing Dropper.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com

EC REP
Obelis s.a
Boulevard General Wabis 53
1030 Brussels, Belgium
Tel: +32) 2 732 59 54, Fax: +32) 2 732 60 03
E-mail : mail@obelis.net

CE
0123

| | |
|------------|------------|
| Revision : | 05 |
| Date : | 23-05-2019 |

LOT Symbol for "BATCH CODE" **U** Symbol for "USE BY" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY"
CAUTION Symbol for "CAUTION" **i** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"
EC REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **DO NOT USE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER NAME & ADDRESS** Symbol for "MANUFACTURER NAME & ADDRESS"
TEMPERATURE LIMITATION Symbol for "TEMPERATURE LIMITATION" **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.