

FUSION I-SEAL

Light Curing Glass Ionomer Composite Cement

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Fusion I-Seal single component, light cured glass ionomer cement for lining, extended fissure sealing and smaller lesions.

Composition : Urethane Dimethacrylate resin, Bisphenol A Glycidyle Methacrylate, Barium Sulphate, FluoroAluminoSilicate Glass Powder, Silanated Barium Glass Powder, Amorphous Fumed Silica, Photoinitiators and Stabilizers

Indications : ♦ Lining under all kinds of filling materials ♦ Extended fissure sealing ♦ Restoration of smaller lesions.

Instructions for use : Thoroughly clean and air dry the cavity. Apply material to the cavity directly with the help of applicator tip. Do not exceed the thickness of 1mm. Light cure for 30- 40 sec using standard curing light and proceed with standard restoration technique.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Contraindications : Fusion I-Seal is contraindicated for use in patients with known allergies to methacrylate or any other if its components.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

Ref : 30002-1

Trial Pack : 1 x 2g Syringe, 5 x Applicator Tips.

Ref : 30002

Intro Pack : 2 x 2g Syringes, 10 x Applicator Tips.

Ref : 30003

Economy Pack : 4 x 2g Syringes, 20 x Applicator Tips.

Ref : 30003-1

Intro Pack : 1 x 3g Syringe, 10 x Applicator Tips.

Ref : 30003-2

Kit Pack : 2 x 3g Syringes, 20 x Applicator Tips.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com

EC REP
Obelis s.a
Boulevard General Wahis 53
1030 Brussels, Belgium
Tel: +32(2) 732 59 54, Fax: +32(2) 732 60 03
E-mail : mail@obelis.net

CE
0123

Revision :	07
Date :	23-05-2019

LOT Symbol for "BATCH CODE" **USE BY** Symbol for "USE BY" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY"
CAUTION Symbol for "CAUTION" **CONSULT INSTRUCTION FOR USE** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"
EC REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **DO NOT USE IF PACKAGE IS DAMAGE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER NAME & ADDRESS** Symbol for "MANUFACTURER NAME & ADDRESS"
TEMPERATURE LIMITATION Symbol for "TEMPERATURE LIMITATION" **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.