

BIOENAMEL

RE-MINERALIZING & DESENSITIZING GEL

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : BioEnamel is a re-mineralizing gel which reduces post whitening sensitivity and effectively enhances the re-mineralization of the enamel.

Composition :

Glycerine, Potassium Nitrate, Bioglass, Carbopol.

Indications :

- ◆ Remineralization of de-mineralized tooth structure.
- ◆ Post whitening sensitivity.
- ◆ Sensitivity caused due to non-cariou lesions and sensitivity after oral prophylaxis.

Instructions for Use :

- ◆ Bio Enamel is presented in a convenient dispenser pen with paint on brush for easy application. The gel will be dispensed on the brush tip by gently rotating bottom of the pen clockwise. First time use requires additional rotations for the gel to appear on the brush tip.
- ◆ Brush your teeth each time before applying BioEnamel. BioEnamel is more effective on clean teeth.
- ◆ Apply a small amount of BioEnamel in a thin layer and cover the entire area under treatment.
- ◆ Avoid lips from contacting teeth for 30 - 60 sec.
- ◆ Leave the BioEnamel on the tooth surface and avoid rinsing, brushing your teeth for 4 to 6 hours.
- ◆ Apply BioEnamel as often as needed.

Caution :

Always close the pen with lid after use to avoid drying of the gel on the brush bristles.

Contra-Indications :

This product is contra-indicated for use in or by individuals who have known hypersensitivity to any of the constituents of the products.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Ref. 90012

Presentation :

1 x 2ml Dispensing Pen.














Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

 **Prevest DenPro Limited**
Unit II, EPIP, Bari Brahmama,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com

EC REP
Obelis s.a
Boulevard General Wahis 53
1030 Brussels, Belgium
Tel: +32 2 732 09 54. Fax: +32 2 732 60 03
E-mail : mail@obelis.net

CE
0123

Revision :	06
Date :	23-05-2019

-  Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"
-  SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.