

Actino Gel/Liquid®

Dental Etching Gel/Liquid

Actino Dental Etching Gel/Liquid is a Blue tinted viscous gel/Liquid of 37% Phosphoric acid for etching tooth enamel and dentine or composite surfaces prior to the bonding of any composites, sealants or bonding adhesives.



Indications

- Used for conditioning both dental enamel and dentine surfaces prior to the placement of an adhesive restoration.

Benefits

- 37% phosphoric acid to give best dentine etching results.
- Provides selective and controlled etching.
- Does not dry etched tooth surface.
- Gel form allows for more convenient usage and controlled application on the tooth surface.

Chemistry

Phosphoric Acid, Methylene Blue Indicator, Glycerine, Purified Water and Amorphous Silica.

Technical Data

- 1) Colour : Blue
- 2) PH : 1-2
- 3) Phosphoric Acid : 37%

Techniques



Cavity Preparation



Placement of Actino Gel and allow the gel to stay for 10 to 15 second.



Rinse thoroughly with water spray



Dry etched tooth surface with clean, dry air / cotton swab.

Presentation

- Ref: 10006-1 (Trial Pack : 1 x 2ml Syringe)
 Ref: 10006-2 (Trial Pack : 1 x 3ml Syringe, 10 Applicator Tips)
 Ref: 10006 (Intro Pack : 1 x 5ml Syringe, 10 Applicator Tips)
 Ref: 10007 (Economy Pack : 4 x 5ml Syringe, 40 Applicator Tips)
 Ref: 10008 (Jumbo Pack : 1 x 50 ml Syringe Actino Gel, 2 x Empty Syringes 5 ml, 1 x 20 Applicator Tips)
 Ref: 10009 (Liquid Packing : 10ml Plastic dropper bottle)

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
 Tel.: 91 1923 222774/222779
 Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
 Website : www.prevestdenpro.com
 E-mail : customer@prevestdenpro.com

Actino Gel/Liquid®

Dental Etching Gel/Liquid



Actino Dental Etching Gel/Liquid is a Blue tinted viscous gel/Liquid of 37% Phosphoric acid for etching tooth enamel and dentine or composite surfaces prior to the bonding of any composites, sealants or bonding adhesives.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com