

ACTINO

DENTAL ETCHING LIQUID

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Actino is a Blue tinted viscous liquid of 37% Phosphoric acid, silica free composition recommended for use on enamel and dentine. Its non drying thixotropic liquid composition prevents thickening and dry out with time and thus maintain its flow characteristics.

Composition : Phosphoric Acid, Methylene Blue Dye, Purified Water and Thickening agent.

Applications : Used for conditioning both dental enamel and dentine surfaces prior to the placement of an adhesive restoration.

Instructions for Use : Clean the area to be etched, rinse and dry thoroughly. Place the tip against the surface to be etched and slowly dispense the liquid on the surface to lay a fine line or paint a thin layer to the areas to be etched. Allow the liquid to stay for 10-15 seconds and rinse thoroughly with water spray. Dry etched tooth surface with clean, dry air.

Precautions : The product contains phosphoric acid. It can burn and cause irritation to the tissues and therefore confine the use to areas to be etched. Avoid contact with eyes and prolonged contact with skin.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Contraindications : The products contains 37% Phosphoric acid, so the product should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

3years under proper storage.

Presentation :

Ref : 10009

10ml Plastic dropper bottle.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com
Made In India



CE
0123

Revision :	06
Date :	24-07-2020

LOT	Symbol for "BATCH CODE"	USE BY	Symbol for "USE BY"	NON STERILE	Symbol for "NON STERILE"	KEEP DRY	Symbol for "KEEP DRY"
CAUTION	Symbol for "CAUTION"	CONSULT INSTRUCTION FOR USE	Symbol for "CONSULT INSTRUCTION FOR USE"	REF	Symbol for "CATALOGUE NUMBER"		
ECI/REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE DO NOT USE IF PACKAGE IS DAMAGE							
KEEP OUT OF THE REACH OF CHILDREN	Symbol for "KEEP OUT OF THE REACH OF CHILDREN"	MANUFACTURER NAME & ADDRESS	Symbol for "MANUFACTURER NAME & ADDRESS"				
TEMPERATURE LIMITATION	Symbol for "TEMPERATURE LIMITATION"	KEEP AWAY FROM SUNLIGHT	Symbol for "KEEP AWAY FROM SUNLIGHT"	CORROSIVE	Symbol for "CORROSIVE"		

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.