

FLORENCE

IN-OFFICE WHITENING SYSTEM

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Florence is a hydrogen peroxide based rapid, In-Office tooth whitening system. Powder and liquid are mixed together to form a freshly activated gel which is applied on the teeth with a fine applicator tip or brush.

Composition :

Liquid contains : 35% Hydrogen Peroxide, Glycerine

Powder contains : Potassium Nitrate, Potassium Hydroxide, Fumed Silica

Gingiva Shield VLC contains : Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Fumed Silica, Barium Glass Powder, Photoinitiator and Stabilizer.

Indications :

Teeth whitening for therapeutic use

Precautions :

- ◆ For professional use only.
- ◆ Patient must wear protective eyewear.
- ◆ Professional must wear gloves, mask and protective eyewear.
- ◆ Do not use on pregnant or lactating woman and children below the age of 15 years.
- ◆ Do not use on any person having known peroxide allergies.
- ◆ Handle the liquid syringes carefully as the same may be pressurized.
- ◆ Do not allow mixed gel or liquid to come in contact with skin, eye and soft tissues.

Indications for use :

- ◆ Determine the pre operative shade for comparison after whitening.
- ◆ Clean the teeth with pumice.
- ◆ Place cheek retractors and then cover exposed lip surface with petroleum jelly.
- ◆ Dry teeth and apply Gingiva Shield VLC to both arches, slightly over lapping enamel and interproximal spaces.
- ◆ Light cure in a fanning motion for 20-30 sec until it is cured.
- ◆ Open powder pot and carefully empty the liquid into the pot.
- ◆ Mix powder & liquid with a brush to form a homogeneous gel.
- ◆ Apply a thick layer of gel to all teeth undergoing treatment.
- ◆ Leave the gel for 10 minutes.
- ◆ Suction off saliva using an aspiration tip.
- ◆ Repeat the procedure 2-3 times using mixed gel.
- ◆ Use the mixed gel within 30-40 minutes after mixing.
- ◆ After the last application, suction all the gel, then wash and apply suction.
- ◆ Remove Gingiva Shield VLC Barrier by lifting it from one end.

Storage Condition : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

Single Patient Kit : Ref. 90001

2 x 0.3g Powder Pots, 2 x 3ml Liquid Bottles, 1 x 1.2ml Gingiva Shield VLC (Gingival Barrier Syringe), 2 x Micro Brushes, 2 x Dispensing Tips

Two Patient Kit : Ref. 90002












4 x 0.3g Powder Pots, 4 x 3ml Liquid Bottles, 2 x 1.2ml Gingiva Shield VLC (Gingival Barrier Syringe), 1 x Mixing Pot, 4 x Micro Brushes, 4 x Dispensing Tips.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com

Revision :	06
Date :	23-05-2019

-  Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"
-  SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.