

24 CARAT

Advanced Zero Sensitivity Tooth Whitening System

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

24 Carat Pure White Smile is a revolutionary, state-of-art Teeth Whitening Gel. Its advanced zero sensitivity formula eliminates sensitivity. The optimized pH gel consists of the active carbamide peroxide suspended in a high tack, high viscosity gel which considerably reduces the treatment time.

Composition :

Carbamide Peroxide, EDTA, Citric Acid, Propylene Glycol, Carbopol, Glycerine. Potassium Nitrate and Spearmint Oil.

Instructions for use :

Remove cap from the syringe and fix dispensing tip. Squeeze whitening gel into the tray in small drops at each tooth site. Do not overfill the tray. Brush and floss teeth thoroughly and seat the tray firmly into teeth. Wipe over flowing excess gel off the gums with a cotton swab or tissue by holding the tray with one hand. Rinse with cold water. Do not swallow rinses. Wear tray for 30 minutes. You may wear for longer period for further results. Remove tray and brush teeth to remove gel. Clean tray with soft brush or with cotton swab and rinse with cold water. Wipe the tray with dry tissue and store in zipper pouch/plastic box. Repeat the process for both upper and lower teeth once a day until teeth have reached desired shade of whiteness. Maximum of twelve applications recommended.

Warning :

For professional use only .

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Cautions :

Only natural teeth can be bleached. Pregnant and lactating women are advised to avoid whitening treatment. Check patient's history for allergies to carbamide peroxide/hydrogen peroxide and other ingredients. If you experience any problem, consult your dentist.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

3Years from the date of manufacturing .

Presentation :

Ref : 90003

Intro Pack : 1 x 5ml Syringe, 16%+ 1 x Dispensing Tips

Ref : 90006

Economy Pack : 4 x 1.5ml Syringes, 16% + 4 x Dispensing Tips

Ref : 90004

Intro Pack : 1 x 5ml Syringe, 22%+ 1 x Dispensing Tips

Ref : 90007

Economy Pack : 4 x 1.5ml Syringes, 22% + 4 x Dispensing Tips

Ref : 90005

Intro Pack : 1 x 5ml Syringe, 35%+ 1 x Dispensing Tips

Ref : 90008

Economy Pack : 4 x 1.5ml Syringes, 35% + 4 x Dispensing Tips

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmna,

Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com

Revision :	05
Date :	23-05-2019

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.