

A.C.E.TM Ceramic Repair Kit

Comprehensive set for intraoral repairs of ceramic and composite restorations. This comprehensive system comprises the necessary bonding and veneering materials.



Indications

- Repair of porcelain fused to metal crown and bridges & inlays & onlays and ceramic veneered restorations.
- All ceramic restorations.

Benefits

- Comprehensive set for intraoral repair of ceramic and composite restorations.
- Due to its high bonding values this system is a reliable assistant in incontinent situations.
- A.C.E. ceramic repair kit comprises the necessary bonding and veneering materials.


Chemistry

- ◆ **Cera-Etch** : 5% Hydrofluoric Acid, purified water and thickening agents.
- ◆ **Silane-X** : Organo functional Silane, Ethyl Alcohol and Acetone.
- ◆ **Fusion Flo** : Bisphenol A Glycidyl Methacrylate, Ethoxlated Bisphenol A Diethoxy Methacrylate, Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Silanated Barium Glass Powder, Amorphous Fumed Silica, Total Inorganic Filler 70%.
- ◆ **Renew Universal** : Functional Monomers Such As 10-MDP and 4-META.


Technical Data

- 1) Curing Time : 30-40 Sec.
- 2) PH : 2-3


Techniques

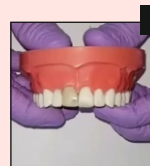


Fractured ceramo metal crown

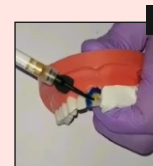

Etch Ceramic only for 90 sec followed by rinsing and dry

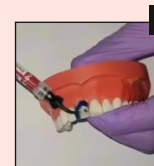


Apply Silane-X with Ceramic & dry for 1 min

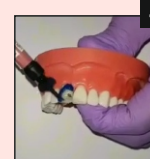


Apply Renew Universal & light cure for 20 sec



Mask the metal with Fusion Flo opaque & light cure for 40 sec

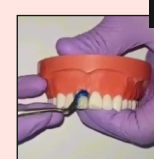

Build up of defect with Fusion Universal & cure for 40 sec

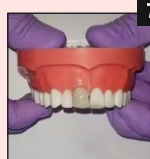

Restored defect of fractured ceramo-metal crown

Presentation

Ref: 50002

Kit Contents : 1 x 1.2ml Syringes **CeraEtch** + 5 Applicator Tips. (Ref : 10012), 1 x 1.2ml Syringes **Silane-X** + 5 Applicator Tips. (Ref : 10014), 1 x 1.2ml Syringes **Fusion Flo**, Opaque + 5 Applicator Tips. (Ref : 20008-2), 1 x 1.2ml Syringe **Renew Universal** + 5 x Applicator Tips (Ref : 10017), 1 x Instruction manual.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com

A.C.E.TM

Ceramic Repair Kit



Comprehensive set for intraoral repairs of ceramic and composite restorations. This comprehensive system comprises the necessary bonding and veneering materials.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer care@prevestdenpro.com