

# Alfina

Regular/ fast set alginate Impression material

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description:** Alfina is a dust free alginate impression material for high precision elastic impressions. It is available in regular and fast set varieties.

**Instructions for use :** Open the bag and pour the alginate into an airtight container. Keep the lot number and expiration date of the product by cutting the relevant details off the bag. To ensure a proper powder-to-water ratio, shake the container 2 or 3 times to disperse the powder before mixing. Take 18g of powder into a rubber mixing bowl and add 36 ml of water to it. Spatulate vigorously till you obtain a smooth glossy cream. Load the mixed material into the impression tray and seat it inside the mouth. Remove the impression after setting and rinse it well. For best results, the model must be poured immediately without any undue delay.

The mixing time, working time and the setting times of the regular and fast set varieties are as below

TIMES FOR CLINICAL USE	REGULAR SET	FAST SET
Mixing Time	45 seconds	45 seconds
Manipulation Time	60 seconds	30 seconds
Setting Time in Mouth	60 seconds	30 seconds
Total Working Time	2 minutes 45 sec	1 minutes 45 sec

**Caution:** Times for clinical use are indexed to the start of mixing with deionized water at 23°C/73°F. Colder water slows setting, warmer water speeds up setting. Variations in water hardness may result in deviations in times. Distilled water is ideal for alginate because it eliminates contamination of the mix with calcium, fluoride, and other materials found in tap water.

**Storage Conditions :** Store the container at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture.

**Shelf Life : 3 years from the date of manufacturing.**

**Presentation :**

Ref : 70009

Alfina Regular/fast set 450gm aluminum foil bag

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : 1800-180-7172  
Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made In India

EC REP  
Obelis s.a  
Boulevard General Wahlen 53  
1030 Brussels, Belgium  
Tel: +1(32) 2 732 59 54, Fax: +1(32) 2 732 69 03  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

CE  
0123

Revision :	06
Date :	24-07-2020

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"

Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.