

ALVOCURE

Non Adherent Periodontal Dressing For Dry Socket

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Alvocure is non adherent periodontal dressing for dry socket. It provides a soothing effect throughout the healing process. Its fibrous consistency allows for easy filling and adherence to the tooth socket after extraction and it is self eliminating.

Composition : Penghawar, Calcium Carbonate, Eugenol and Excipients.

Intended Purpose/Intended Use : It is intended to be used as a periodontal dressing (dry socket)

Indications for use :

It is used as a non-adherent periodontal dressing in dry sockets.

Instructions for use :

Local use only. A small pellet of Alvocure should be applied aseptically to cover the socket after verification and cleaning the socket. Do not suture. The dispensed material shall not be placed back in the container and the dose once applied or used shall not be reused.

Warnings :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton soaked in alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Precautions :

1. Wear suitable gloves for application.
2. The cap should always be replaced immediately after use.

Contraindications :

Do not use in patients if allergic to any of its ingredients and on deciduous teeth (children under 12 years of age)

Storage Conditions :

Store at temperature between 10°C (50°F) to 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool and dry place.

Shelf Life :

3 years from the date of manufacturing.

Presentation :

Ref : 40023

Jar of 12g

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India



Revision :	09
Date :	30-09-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE" Symbol for "USE-BY DATE" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC/REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

Symbol for "DO NOT USE IF PACKAGE IS DAMAGED" Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER"

Symbol for "TEMPERATURE LIMIT" Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.