

CAL LC

Light Cured Radiopaque Calcium Hydroxide Paste



Product Description : CAL LC is light cured radiopaque Calcium Hydroxide Paste useful in pulp capping and as a protective base/liner under dental filling materials, cements and other base materials.

Composition : Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Silanated Barium Glass, Amorphous Fumed Silica, Barium Sulphate, Calcium Hydroxide.

Indications : Direct pulp capping, lining under all filling materials, temporary and long term sealing.

Instructions for use : Thoroughly clean and air dry the cavity. Apply material to the cavity directly with the help of applicator tip. Do not exceed the thickness of 1mm. Light cure for 30-40 sec using standard curing light and proceed with standard restoration technique.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Contraindications :

CAL LC is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins and also for direct application to dental pulp tissue.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 Years from the date of manufacturing.

Presentation :

Ref: 30004

Trial Pack : 1 x 2g Syringe + 5 x Applicator Tips.


Ref: 30005

Intro Pack : 2 x 2g Syringes + 10 x Applicator Tips.

Ref: 30005-1

Economy Pack : 4 x 2g Syringes + 20 x Applicator Tips.





Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.


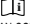

 **Prevest DenPro Limited**
Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India




Obelis s.a
Boulevard General Wahlen 53
1030 Brussels, Belgium
Tel: + (32) 2 732 59 54, Fax: + (32) 2 732 60 03
E-mail: mail@obelis.net

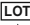

CE
0123



Revision :	07
Date :	24-07-2020

 **LOT** Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.