

**Product Description :** This is premixed oil based calcium hydroxide paste containing iodoform. The addition of iodoform to calcium hydroxide improves radio-opacity and improves disinfecting property of the paste. Non Hardening paste, solubilized calcium hydroxide remains active in root canal for a longer time.

**Composition :** Calcium Hydroxide, Iodoform, Silicon Oil.

**Indications :** ♦ Disinfects canals ♦ Inhibits bacteria growth ♦ Promotes periapical healing ♦ Stimulates apexification and apexogenesis ♦ Temporary filling material for infected root canals ♦ Treatment of root resorption.

**Instructions for Use :** After root canal preparation, clean and dry root canal. Attach disposable needle to the syringe and check the flow of paste through the needle. Insert needle into the canal, about 2mm short of the apex. Fill canal completely by pushing the plunger while slowly withdrawing the syringe. Clean excess paste with a sterile cotton pellet.

**Warning :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Storage Conditions :** Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf life :** 3 years from date of manufacturing.

**Presentation :**

Ref : 40005

**Trial Pack :** 1 x 2g Syringe, 5 x Applicator Tips.

Ref : 40005-1

**Intro Pack :** 1 x 3g Syringe, 10 x Applicator Tips.

Ref : 40006

**Economy Pack :** 4 x 2g Syringes, 20 x Applicator Tips.

**Disposal conditions :**

To be disposed off based on the prevailing rules of the country in which it is used.

### Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,  
Jammu-181133, India.














www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com

Made In India

Revision :	07
Date :	24-07-2020

 <b>LOT</b> Symbol for "BATCH CODE"	 Symbol for "USE BY"	 Symbol for "NON STERILE"	 Symbol for "KEEP DRY"
 Symbol for "CAUTION"	 Symbol for "CONSULT INSTRUCTION FOR USE"	 Symbol for "CATALOGUE NUMBER"	
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE			
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"			
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"			
 Symbol for "MANUFACTURER NAME & ADDRESS"			
 Symbol for "TEMPERATURE LIMITATION"			
 Symbol for "KEEP AWAY FROM SUNLIGHT"			

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.