

CAMPHENOL

CAMPHORATED PARACHLOROPHENOL

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : A solution for root canal cleansing and disinfection. Due to its oily composition, Camphenol remains active for long time.

Composition :

Parachlorophenol and Camphor

Intended Purpose / Intended Use: Root canal cleansing and disinfection.

Indications for Use:

1. Disinfection of the dentine after cavity preparations.
2. Disinfection after pulpectomy and pulp dressing. Treatment of post traumatic inflammation.
3. Disinfection of the root canal.

Instructions for use :

Place a cotton wool pledged soaked in camphenol into the pulp chamber and seal. Remove the cotton wool dressing after a week's time and restore the tooth. The dispensed material shall not be placed back in container and the dose once applied and used shall not reused.

Warnings:

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions :

Avoid contact with skin and eyes.

Contraindications:

Do not use in patients if allergic to any of its components.

Storage conditions :

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life :

3 years from the date of manufacturing.

Presentation :

Ref. 40015

1 x 15ml Glass Bottle, 1 x Dispensing Dropper.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

Revision :	09
Date :	07-12-2023

 Symbol for "UNIQUE DEVICE IDENTIFIER"




 Symbol for "MEDICAL DEVICE"  Symbol for "DANGER"

 Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "CORROSIVE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.