

COPAL-F

CAVITY VARNISH WITH FLUORIDE

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Copal-F is a Copal Gum based protective varnish for the insulation of the exposed dentine, reduces penetration of fluids into the tooth and subsequent placed restoration. Addition of Sodium Fluoride provides a thermal barrier for deep restoration. Alleviates thermal sensitivity. Needs no thinners.

Composition : Copal-Gum, Iso Propyl Alcohol, and Sodium Fluoride.

Indications :

Copal-F is the ideal cavity liner and dentinal tubuli sealer. It assures perfect protection to the exposed surface of dentine in teeth prepared for permanent fillings or restorations and is useful as an immediately varnish prior to the insertion of amalgams, cements or composites.

Instructions for Use :

Prepare the cavity, Put small quantity of cavity varnish on a cotton pellet and apply on the inner walls of cavity. Allow to dry for 40 seconds. Re-apply another layer if necessary. Use a new cotton pellet for each application. It is recommended that two thin layers be applied instead of one thick layer.

Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention.

If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref : 40027

1 x 15ml Glass Bottle, 1 x Dispensing Dropper

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India

EC REP

Obelis s.a

Boulevard General Wahls 53

1030 Brussels, Belgium

Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03

E-mail : mail@obelis.net

CE
0123

Revision :	06
Date :	24-07-2020

- Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"
- Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"
- SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "FLAMMABLE"
- Symbol for "HARMFUL IRRITANT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.