

DOLO

EDTA GEL FOR ROOT CANALS

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Dolo EDTA Gel is 17% EDTA with 10% carbamide peroxide in water soluble gel base for use in chemical and mechanical cleansing of root canal preparation. It lubricates the root canal, chelates the calcium salts to facilitate endodontic instrument penetrations. Due to its effervescent action with sodium hypochlorite solution, cleaning of root canal becomes very easy. The use of sodium hypochlorite with the gel promotes internal bleaching of the tooth. This process is enhanced by the release of oxygen from carbamide peroxide.

Composition : EDTA, Carbamide Peroxide, Glycerine and Polyethylene Glycol.

Indications :

For chemical & mechanical preparation of root canals. Cleansing and enlarging of calcified root canals.

Instructions for Use :

Apply Dolo EDTA Gel at the canal opening with a pellet or coat the instrument to be used with the gel for lubrication. Fill and ream canal by conventional method, Irrigate with 3% Sodium Hypochlorite solution to remove the debris from the canal. Rinse the canal with water to remove the EDTA Gel. Dry Canal with absorbent paper points before filling.

Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions : Avoid injecting solution beyond apex of canal.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life :

3 years from the date of manufacturing.

Presentation :

Ref: 40002

Trial Pack : 1 x 2g Syringe, 5 x Applicator Tips.

Ref: 40002-1

Intro Pack : 1 x 3g Syringe, 10 x Applicator Tips.

Ref: 40003

Economy Pack : 4 x 2g Syringes, 20 x Applicator Tips.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com
Made In India

EC REP

Obellis s.a

Boulevard General Wahiis 53

1030 Brussels, Belgium

Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03

E-mail : mail@obellis.net

CE
0123

Revision : 09

Date : 24-07-2020

- Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"
- Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"
- SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.