

ENDOSEAL

ROOT CANAL OBTURATION CEMENT

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Endoseal is a radiopaque, non-resorbable, Zinc Oxide-Eugenol based root canal sealer. Its composition is well tolerated by the tissues and provides anti inflammatory action.

Composition :

Powder contains : Zinc Oxide, Barium Sulphate, Thymol Iodide, Magnesium Stearate.

Liquid Contains : Eugenol & excipients.

Indications : Root canal filling and sealing in association with a gutta percha point.

Instructions for Use : Mix carefully for approximately 45-60 seconds, one scoop of powder and five drops of liquid to obtain a thick mix, which can easily be introduced into the canal with a suitable root canal filler, or in conjunction with a silver, acrylic or gutta percha point. Net setting time is 45-60 minutes from start of mix.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions : Do not use Endoseal with patients that have a history of allergic reaction to iodine compounds. Avoids contact with eyes.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life :

3 years from the date of manufacturing.

Presentation :

Ref : 40031

15g Powder, 10ml Liquid, 1 x Dispensing Dropper, 1 x Spoon and 1 x Mixing Pad.

Ref : 40031-1





20g Powder, 10ml Liquid, 1 x Dispensing Dropper, 1 x Spoon and 1 x Mixing Pad.




Disposal conditions :

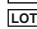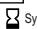

To be disposed off based on the prevailing rules of the country in which it is used.



Prevest DenPro Limited
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India



Revision :	08
Date :	24-07-2020

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.