

IFU Size : 161 H x 104 W

## EpoSolv



### Solvent For Resin & Phenolic Compounds

**Product Description :** EpoSolv solvent is used for removing for resin sealers and phenolic compounds at the time of root canal re-treatment.

**Composition :** Formamide and Phenylethanol.

**Intended Purpose / Intended Use :** It is intended to be used for softening and dissolving root canal filling in endo retreatment cases.

**Indications for Use :** EpoSolv is a solvent for epoxy and other resin sealers during endodontic re-treatment. EpoSolv aids in the softening of phenolic resin type fillings. EpoSolv does not provoke excessive salivation if it comes into contact with the mucosa.

#### Instructions for Use :

**Recommended one visit method :** Clean the pulp-chamber and entrance to the canal. Place a drop of EpoSolv into the chamber using a micro applicator tip. Entry into the canal should be done by once dipping the point of the instrument in EpoSolv before each application. The instrument must be removed from the canal as soon as a certain resistance is felt, then wipe and repeat the operation a number of times, until reaching the apex.

**Recommended two visit method :** (For the removal of the root canal filling material at a subsequent visit).

**On the initial visit :** By mechanical means, clean the pulp-chamber and entrance to the canal. Place into the cavity a small cotton pellet soaked in EpoSolv. Compress slightly. Fill with a sealing cement. Re-open either the next day or some days later. The paste will have disintegrated and the mechanical reaming will present no difficulty. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hyper sensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :** A tooth treated with EpoSolv must be carefully dried before a new restoration made of eugenol resin or acrylic, is placed.

#### Contraindications :

1. The product should not be used in patients with history of allergic reaction to any of the component.
2. The product should not be used beyond the confinements of root canal.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf life :** 3 years from the date of manufacturing.

#### Presentation :

Ref : 40035

1 x 15ml Glass Bottle, 1 x Dispensing Dropper.

**Disposal Conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park,  
(EPIP), Bari Brahmmana, Jammu-181133,  
India. www.prevestdenpro.com  
**Customer Care No. :** + 91 8999074151  
**Email :** customercare@prevestdenpro.com  
Made in India

EC REP  
Obelis s.a  
Boulevard General Wehiss 53  
1050 Brussels, Belgium  
Tel: +32 2 732 99 94 Fax: +32 2 732 60 03  
E-mail: mail@obelis.net



Revision : 06  
Date : 10-02-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

MD Symbol for "MEDICAL DEVICE"

X Symbol for "HARMFUL IRRITANT"

LOT Symbol for "BATCH CODE"    U Symbol for "USE-BY DATE"    N Sterile Symbol for "NON STERILE"    Drip Symbol for "KEEP DRY"  
Caution Symbol for "CAUTION"    I Symbol for "CONSULT INSTRUCTION FOR USE"    REF Symbol for "CATALOGUE NUMBER"  
EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"  
No Fire Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"    No Children Symbol for "KEEP OUT OF THE REACH OF CHILDREN"    Manufacturer Symbol for "MANUFACTURER"  
T Symbol for "TEMPERATURE LIMIT"    Sunlight Symbol for "KEEP AWAY FROM SUNLIGHT"    DANGER Symbol for "DANGER"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.