

# EUGENOL

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Reacts with zinc oxide to form a temporary cement for sedative dental dressing or lining under non- composite resin restoration.

**Composition :** Eugenol.

**Intended Purpose/Intended Use :** It is intended to be used as a temporary filling or dressing material.

**Indications for Use :** In combination with zinc oxide to be used as temporary filling or as dressing under non composite dental restorations.

**Instructions for Use :** Place a drop on a cotton bud and gently apply to the inflamed area. Mix with Zinc Oxide to form a thick paste and place into a tooth cavity either as a dressing on a painful tooth or as a lining under a non composite resin restoration. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

**Precautions :** Avoid contact with eyes. Inhibits the polymerization of composite resin and should not be used as a dressing prior to placing a composite resin restoration.

**Contraindications :** It is contraindicated for use in patient who have a history of severe allergic reaction to Eugenol.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

**Presentation :**

Ref : 40016

1 x 15ml Glass Bottle, 1 x Dispensing Dropper

Ref : 40017

1 x 30ml Glass Bottle, 1 x Dispensing Dropper

Ref : 40018

1 x 110ml Glass Bottle, 1 x Dispensing Dropper

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.



**Prevest DenPro Limited**

Unit II, Export Promotion Industrial Park,  
(EPIP), Bari Brahmna, Jammu-181133,  
India. [www.prevestdenpro.com](http://www.prevestdenpro.com)

**Customer Care No. :** + 91 8899074151

**Email :** [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)

Made in India

<b>Revision :</b>	10
<b>Date :</b>	25-09-2023

Symbol for "UNIQUE DEVICE IDENTIFIER"

Symbol for "BATCH CODE" Symbol for "USE-BY DATE" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER"

Symbol for "TEMPERATURE LIMIT" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.