

FLUORODIP BIOACTIVE

VARNISH FOR DESENSITIZING & RE-MINERALIZATION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

FluoroDip Bioactive is a sodium fluoride containing dental desensitizing varnish fortified with bio-glass for enhanced remineralization of enamel and dentine. FluoroDip Bioactive is an alcoholic suspension of sodium fluoride and bio-glass in natural shellac with artificial flavoring. 1 ml of FluoroDip Bioactive varnish contains 50mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Composition : Sodium Fluoride, Re-mineralizing Agent, Rosin Gum, Solvent and Flavouring Matter.

Intended Purpose/Intended Use :

It is used in Desensitizing and Re-mineralization of enamel and dentine.

Indications for Use :

It is used in :

- ♦ Treatment of exposed and sensitive root surfaces
- ♦ Treatment of hypersensitive teeth

Instructions for Use :

- ♦ FluoroDip Bioactive Varnish is presented in mono dose vials.
- ♦ Thoroughly clean tartar from treatment area and dry the area with cotton pellets.
- ♦ Uniformly apply FluoroDip Bioactive varnish with the micro brush as a thin film covering the entire area under treatment. Gently air dry the varnish for 10 to 20 sec and slightly moisture the area of application to ensure proper setting of the varnish.
- ♦ The patient should be advised not to brush for a minimum 4 to 6 hours after the treatment.

Warning :

- ♦ The product contain substances that may cause allergic reaction by skin contact in certain individuals. If prolonged contact with oral soft tissue occurs flush with plenty amount of water.
- ♦ If allergic reaction occurs, seek medical attention as needed, discontinue the use if required.

Precautions :

- ♦ Avoid use of this product in patients with known allergies.
- ♦ Avoid brushing of teeth for at least four hours after the application.
- ♦ Use protective gloves when using the product.

Contraindications :

This product is contraindicated for use in individuals who have shown hypersensitivity to fluoride and colophony.

Storage Conditions : Store at temperature between 10°C (50°F) and 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 Years from the date of manufacturing.

Presentation :

Ref : 80001 - 2

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes, 10 x Mixing Wells. (Spearmint Flavour)

Ref : 80001 - 3

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes, 10 x Mixing Wells. (Strawberry Flavour)

Ref : 80001 - 4

1 x 0.5 ml Monodose Vial, 1 x Applicator Brush, (Spearmint Flavour)

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.



Prevest DenPro Limited

Unit II, EPIP, Bari Brahmanna,
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com

Made in India

E C R E P

Obelis s.a

Boulevard General Wauters 53
1030 Brussels, Belgium

Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03

E-mail : mail@obelis.net

CE
0123

Revision :	04
Date :	01-02-2023

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

- Symbol for "BATCH CODE"
 Symbol for "USE-BY DATE"
 Symbol for "NON STERILE"
 Symbol for "KEEP DRY"
 Symbol for "CAUTION"
 Symbol for "CONSULT INSTRUCTION FOR USE"
 Symbol for "CATALOGUE NUMBER"
 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"
 Symbol for "FLAMMABLE"
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
 Symbol for "MANUFACTURER"
 Symbol for "TEMPERATURE LIMIT"
 Symbol for "KEEP AWAY FROM SUNLIGHT"
 Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaint and reporting of unexpected events contact customer care.