

# FLORENCE

## IN-OFFICE WHITENING SYSTEM

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Florence is a hydrogen peroxide based rapid, In-Office tooth whitening system. Powder and liquid are mixed together to form a freshly activated gel which is applied on the teeth with a fine applicator tip or brush.

### Composition :

**Liquid contains :** 35% Hydrogen Peroxide & Excipients.

**Powder contains :** Fillers & Desensitizing Agents.

**Gingiva Shield VLC contains :** Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Fillers, Radiopacifier, Photoinitiator and Stabilizer.

**Intended Purpose / Intended Use :** It is intended to be used for tooth whitening.

**Indications for Use :** Assists in whitening of discolored teeth related to medication, age dependent changes, food related stains.

### Instructions for Use :

- ◆ Determine the pre operative shade for comparison after whitening.
- ◆ Clean the teeth with pumice.
- ◆ Place cheek retractors and then cover exposed lip surface with petroleum jelly.
- ◆ Dry teeth and apply Gingiva Shield VLC to both arches, slightly over lapping enamel and interproximal spaces.
- ◆ Light cure in a fanning motion for 20-30 sec until it is cured.
- ◆ Open powder pot and carefully empty the liquid into the pot.
- ◆ Mix powder & liquid with a brush to form a homogeneous gel.
- ◆ Apply a thick layer of gel to all teeth undergoing treatment.
- ◆ Leave the gel for 10 minutes.
- ◆ Suction off saliva using an aspiration tip.
- ◆ Repeat the procedure 2-3 times using mixed gel.
- ◆ Use the mixed gel within 30-40 minutes after mixing.
- ◆ After the last application, suction all the gel, then wash and apply suction.
- ◆ Remove Gingiva Shield VLC Barrier by lifting it from one end.
- ◆ The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Products containing peroxide are not recommended for use by children under 12 years of age. Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

### Precautions :

- ◆ For professional use only.
- ◆ Patient must wear protective eyewear.
- ◆ Professional must wear gloves, mask and protective eyewear.
- ◆ Do not use on pregnant or lactating woman and children below the age of 15 years.
- ◆ Do not use on any person having known peroxide allergies.
- ◆ Handle the liquid syringes carefully as the same may be pressurized.
- ◆ Do not allow mixed gel or liquid to come in contact with skin, eye and soft tissues.

### Contraindications :

- ◆ Do not use in patients with unknown history of allergy to any of its components.
- ◆ Do not use in pregnant or lactating women.

**Storage Condition :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

### Presentation :

**Single Patient Kit :** Ref. 90001

2 x 0.3g Powder Pots, 2 x 3ml Liquid Bottles, 1 x 1.2ml Gingiva Shield VLC (Gingival Barrier Syringe), 2 x Micro Brushes, 2 x Dispensing Tips

**Two Patient Kit :** Ref. 90002

4 x 0.3g Powder Pots, 4 x 3ml Liquid Bottles, 2 x 1.2ml Gingiva Shield VLC (Gingival Barrier Syringe), 1 x Mixing Pot, 4 x Micro Brushes, 4 x Dispensing Tips.

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

### Prevest DenPro Limited

Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com

Made in India

Revision :	08
Date :	27-01-2023

**UDI** Symbol for "UNIQUE DEVICE IDENTIFIER"

**LOT** Symbol for "BATCH CODE" **USE-BY DATE** Symbol for "USE-BY DATE" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY"

**CAUTION** Symbol for "CAUTION" **CONSULT INSTRUCTION FOR USE** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

**EC REF** Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

**KEEP OUT OF THE REACH OF CHILDREN** Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER** Symbol for "MANUFACTURER" **DO NOT USE IF PACKAGE IS DAMAGE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

**TEMPERATURE LIMIT** Symbol for "TEMPERATURE LIMIT" **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For Feedback, Complaints and reporting of unexpected events contact customer care.