

IFU Size : 161 H x 104 W

FLUORODIP BIOACTIVE

VARNISH FOR DESENSITIZING & RE-MINERALIZATION



Product Description : FluoroDip Bioactive is a sodium fluoride containing dental desensitizing varnish fortified with bio-glass for enhanced remineralization of enamel and dentine. FluoroDip Bioactive is an alcoholic suspension of sodium fluoride and bio-glass in natural shellac with artificial flavoring. It contains 5% NaF equivalent to 2.26% of Fluoride (22600ppm).

Composition : Sodium Fluoride, Bio-glass, Rosin Gum, Ethyl Alcohol, Spearmint Oil.

Intended Purpose / Intended Use : It is used in desensitizing and re-mineralization of enamel and dentine.

Indications for Use:

◆ Desensitizing and Re-mineralization of enamel and dentine.

Instructions for Use :

- ◆ FluoroDip Bioactive Varnish is presented in dropper bottle. Shake the bottle well before use.
- ◆ Thoroughly clean tartar from treatment area and dry the area with cotton pellets.
- ◆ Uniformly apply FluoroDip Bioactive varnish with the microbrush as a thin film covering the entire area under treatment. Gently air dry the varnish for 10 to 20 sec and slightly moisture the area of application to ensure proper setting of the varnish.
- ◆ The patient should be advised not to brush for a minimum 4 to 6 hours after the treatment.

Warnings : FluoroDip Bioactive composition is highly flammable. Handle with care and keep away from open flames.

Precautions :

- ◆ Avoid use of this product in patients with known allergies.
- ◆ Avoid brushing of teeth for at least four hours after the application.
- ◆ Use protective gloves when using the product.

Contra-indications : This product is contra-indicated for use in or by individuals who have shown hypersensitivity to fluoride and colophony.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

Ref : 80001

1 x 10ml Dropper Bottle (Spearmint Flavour)

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

EC REP
Obelis s.a
Boulevard General Wafis 53
1030 Brussels, Belgium
Tel: +32 2 732 99 94, Fax: +32 2 732 00 03
E-mail : mail@obelis.net

CE
0123
Revision : 09
Date : 19-02-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"
LOT Symbol for "BATCH CODE" USE-BY DATE Symbol for "USE-BY DATE" NON STERILE Symbol for "NON STERILE" KEEP DRY Symbol for "KEEP DRY"
CAUTION Symbol for "CAUTION" CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE" REF Symbol for "CATALOGUE NUMBER"
EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION" FLAMMABLE Symbol for "FLAMMABLE"
DO NOT USE IF PACKAGE IS DAMAGE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" MANUFACTURER Symbol for "MANUFACTURER"
TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT" KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT" MEDICAL DEVICE Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaint and reporting of unexpected events contact customer care.