

FLUORODIP BIOACTIVE

VARNISH FOR DESENSITIZING & RE-MINERALIZATION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

FluoroDip Bioactive is a sodium fluoride containing dental desensitizing varnish fortified with bio-glass for enhanced remineralization of enamel and dentine. FluoroDip Bioactive is an alcoholic suspension of sodium fluoride and bio-glass in natural shellac with artificial flavoring. 1 ml of FluoroDip Bioactive varnish contains 50mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Composition : Sodium Fluoride, Bio-glass, Rosin Gum, Ethyl Alcohol and Flavouring Matter.

Indications :

◆Desensitizing and Re-mineralization of enamel and dentine.

Instructions for Use :

◆FluoroDip Bioactive Varnish is presented in mono dose vials.

◆Thoroughly clean tartar from treatment area and dry the area with cotton pellets.

◆Uniformly apply FluoroDip Bioactive varnish with the microbrush as a thin film covering the entire area under treatment. Gently air dry the varnish for 10 to 20 sec and slightly moisture the area of application to ensure proper setting of the varnish.

◆The patient should be advised not to brush for a minimum 4 to 6 hours after the treatment.

Warning : FluoroDip Bioactive composition is highly flammable. Handle with care and keep away from open flames.

Contra-indications :

This product is contra-indicated for use in or by individuals who have shown hypersensitivity to fluoride and colophony.

Storage Conditions : Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 Years from the date of manufacturing.

Presentation :

Ref: 80001 - 2

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes, 10 x Mixing Wells. (Spearmint Flavour)

Ref: 80001 - 3

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes, 10 x Mixing Wells. (Strawberry Flavour)

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.





www.prevestdenpro.com




Customer Care No. : 1800-180-7172

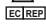

Email : customercare@prevestdenpro.com



CE




Revision :	02
Date :	23-05-2019

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "FLAMMABLE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.