

IFU Size : 161 H x 104 W

## FLUORODIP BIOACTIVE

VARNISH FOR DESENSITIZING & RE-MINERALIZATION



### Product Description :

FluoroDip Bioactive is a sodium fluoride containing dental desensitizing varnish fortified with bio-glass for enhanced remineralization of enamel and dentine. FluoroDip Bioactive is an alcoholic suspension of sodium fluoride and bio-glass in natural shellac with artificial flavoring. 1 ml of FluoroDip Bioactive varnish contains 50mg of sodium fluoride equivalent to 22.6 mg of fluoride.

**Composition :** Sodium Fluoride, Re-mineralizing Agent, Rosin Gum, Solvent and Flavouring Matter.

### Intended Purpose/Intended Use :

It is used in Desensitizing and Re-mineralization of enamel and dentine.

### Indications for Use :

It is used in :

- ◆ Treatment of exposed and sensitive root surfaces
- ◆ Treatment of hypersensitive teeth

### Instructions for Use :

- ◆ FluoroDip Bioactive Varnish is presented in mono dose vials.
- ◆ Thoroughly clean tartar from treatment area and dry the area with cotton pellets.
- ◆ Uniformly apply FluoroDip Bioactive varnish with the micro brush as a thin film covering the entire area under treatment. Gently air dry the varnish for 10 to 20 sec and slightly moisture the area of application to ensure proper setting of the varnish.
- ◆ The patient should be advised not to brush for a minimum 4 to 6 hours after the treatment.

### Warnings :

- ◆ The product contain substances that may cause allergic reaction by skin contact in certain individuals. If prolonged contact with oral soft tissue occurs flush with plenty amount of water.
- ◆ If allergic reaction occurs, seek medical attention as needed, discontinue the use if required.

### Precautions :

- ◆ Avoid use of this product in patients with known allergies.
- ◆ Avoid brushing of teeth for at least four hours after the application.
- ◆ Use protective gloves when using the product.

### Contraindications :

This product is contraindicated for use in individuals who have shown hypersensitivity to fluoride and colophony.

**Storage Conditions :** Store at temperature between 10°C (50°F) and 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

### Presentation :

Ref : 80001 - 2

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes (Spearment Flavour )

Ref : 80001 - 3

10 x 0.5 ml Monodose Vials, 10 x Applicator Brushes (Strawberry Flavour )

Ref : 80001 - 4

1 x 0.5 ml Monodose Vial, 1 x Applicator Brush (Spearment Flavour )

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park  
(EPIP), Bari Brahmna, Jammu-181133,  
India. [www.prevestdenpro.com](http://www.prevestdenpro.com)  
**Customer Care No. :** + 91 8899074151  
**Email :** [customer@prevestdenpro.com](mailto:customer@prevestdenpro.com)  
Made in India

EC REP  
Obelis s.a  
Boulevard General Wahlen 53  
1030 Brussels, Belgium  
Tel: +32 2 732 59 54, Fax: +32 2 732 60 03  
E-mail: [mail@obelis.net](mailto:mail@obelis.net)

CE  
0123

Revision :	05
Date :	06-10-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"    USE-BY DATE Symbol for "USE-BY DATE"    NON STERILE Symbol for "NON STERILE"    KEEP DRY Symbol for "KEEP DRY"  
CAUTION Symbol for "CAUTION"    CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE"    REF Symbol for "CATALOGUE NUMBER"  
EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"    FLAMMABLE Symbol for "FLAMMABLE"  
DO NOT USE IF PACKAGE IS DAMAGE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"    KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN"    MANUFACTURER Symbol for "MANUFACTURER"  
TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT"    KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT"    MD Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaint and reporting of unexpected events contact customer care.