

# FORMACRESOL

**PREVEST**DenPro®  
THE FUTURE OF DENTISTRY

## Product Description :

A solution for root canal cleansing and disinfection. Due to its antibacterial activity, Formacresol is efficient in vital pulpotomy and for the treatment of severely infected root canals.

## Composition :

Mata Para Cresol, Formaldehyde, Glycerine and Iso Propyl Alcohol.

## Indications :

Root canal cleansing

Root canal dressing.

## Instructions for use :

Place a cotton wool pledged soaked in Formacresol into the pulp chamber and seal. Remove the cotton wool dressing after a week time and restore the tooth.

## Warning :

Contains formaldehyde which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

**Contraindications :** It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :** Avoid contact with skin and eyes.

## Storage conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

## Presentation :

Ref: 40014

1x15ml Glass Bottle, Dispensing Dropper.

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : 1800-180-7172  
Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made In India

Revision :	07
Date :	21-07-2020



Symbol for "DANGER"

**LOT** Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

**EC REP** SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"

Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "CORROSIVE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.