

# FUSION FLO SE

Light Cured, Self Adhesive  
Hybrid Flowable Composite

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Fusion Flo SE is a self etch, self adhesive an excellent flowable composite that delivers high strength & wear resistance. It is characterized by optimum surface affinity, penetrates into most difficult area. It combines esthetics as well as flow on demand handling for use. The flowable composite is ready for use and doesn't require the prequisite preparatory steps of etching, priming and bonding.

**Composition :** Bisphenol A Glycidyl Methacrylate, Ethoxlated Bisphenol A Dimethacrylate, Urethane Dimethacrylate, acidic phosphate monomer, Triethylene Glycol Dimethacrylate, Silanated Barium Glass Powder, Amorphous Fumed Silica, Total inorganic filler 70%, 2-Dimethylamino Ethyl Methacrylate, 10-Metharyloyloxydecry Dihydrogen Phosphate, Ethyl 4 Dimethyl Amino Benzoate, Photo Polymerizing agents, Pigment color.

**Intended purpose/Intended use :** It is used as a permanent filling material for anterior and posterior teeth.

**Indications for use :** It is indicated for direct filling of class V (Cervical Caries), root erosions, wedge shaped defects. Fillings of anterior teeth (Class III & IV). Small fillings in posterior teeth.

**Instructions for use :** Apply the material with applicator tip directly in the cavity and pre contour with a suitable instrument and wait for 10 seconds. The layer should not be thicker than 2mm. Light cure for 20-40 sec using standard curing light. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**NOTE :** SELECTIVE ETCHING TECHNIQUE IS HOWEVER RECOMMENDED WHEN ENAMEL SURFACES ARE INVOLVED.

**Warnings :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation and hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

**Precautions :** Eugenol containing materials should not be used with this product because they may effect polymerization and interfere with the hardening process. Being sensitive to heat and light, the syringes should be kept away from direct sun light.

**Contraindications :** It is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins.

**Storage Conditions :** Store at temperature between 10°C (50°F) to 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

### Shelf Life :

3 years from the date of manufacturing.

### Presentation :

Ref. : 20008-3

**Trial Pack :** 1 x 2g Syringes + 5 Applicator Tips

Ref. : 20008-4

**Intro Pack :** 2 x 2g Syringes + 10 Applicator Tips

Ref. : 20008-5

**Intro Pack :** 2 x 2g Syringes + 10 Applicator Tips

**Shades :** A1, A2, A3, A3.5, B1, B2

### Disposal Conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.  
www.prevestdenpro.com  
Customer Care No. : +91 8899074151  
Email : customercare@prevestdenpro.com  
Made in India

EC REP  
Obelis s.a  
Boulevard General Wahis 53  
1030 Brussels, Belgium  
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 69 03  
E-mail : mail@obelis.net

CE  
0123

Revision : 04  
Date : 02-12-2022

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints & reporting of unexpected events contact customer care.