


Fusion Ultra D/C®

Dual Curing Resin Based Luting Cement

Fusion Ultra D/C® is a low viscosity, light and chemically cured, radiopaque, two component nano hybrid luting cement for adhesive luting of indirect composite, zirconium core, metal and porcelain/ceramic restorations to both dentine and enamel.

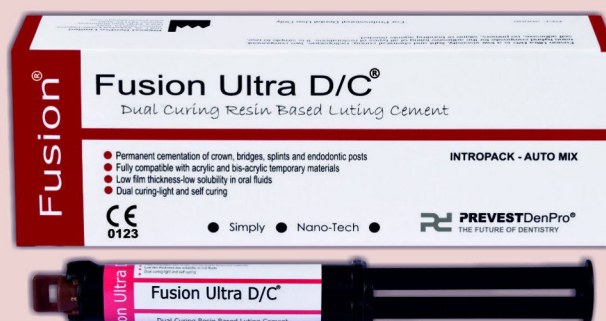

Indications

- Perfect cementation of metal as well as non-metallic posts.
- Permanent cementation of crown, bridges and splints
- Permanent cementation of implant retained crowns

Benefits

- Dual curing - light and self curing (Initial set time 2-2:30 minutes, final set time 3-3:30 minutes)
- Perfect fitting due to low film thickness <30µm
- Low solubility
- Fully compatible with Acrylic and Bis-Acrylic temporary material
- 2 stage polymerization allows fast and easy clean without compromising the working time


Chemistry

Fusion Ultra D/C is a blend of polymerisable resins, difunctional monomers, silanated submicron barium borosilicate glass, surface modified nano amorphous silica, curing initiators, stabilizers in two component-base catalyst pastes.

Technical Data

- 1) Working Time : 60 sec.
- 2) Setting Time : 3 - 4 min.
- 3) Compressive Strength : 195 - 200 MPa.
- 4) Flexural Strength : 90 MPa.
- 5) Film Thickness : 25 µm

Application Technique

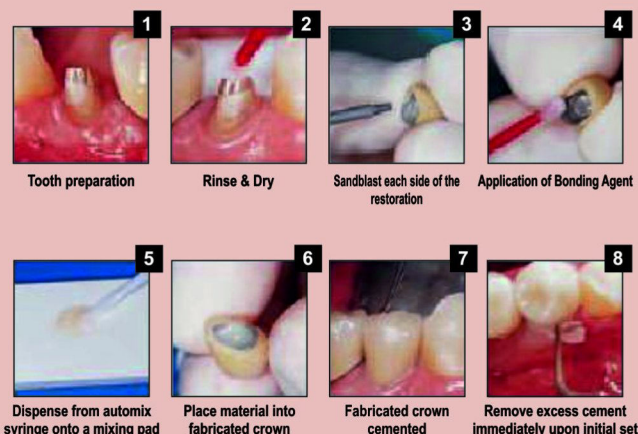

Presentation

Intro Pack :

Ref. : 30006

1 x 9g Automix Syringe, 10 x Automix Tips,
1 x Instruction Manual

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548

Tel.: 91 1923 222774/222779

Toll Free No. : 1800-180-7172


E-mail : info@prevestdenpro.com

Website : www.prevestdenpro.com

E-mail : customercare@prevestdenpro.com

Fusion Ultra D/C®

Dual Curing Resin Based Luting Cement

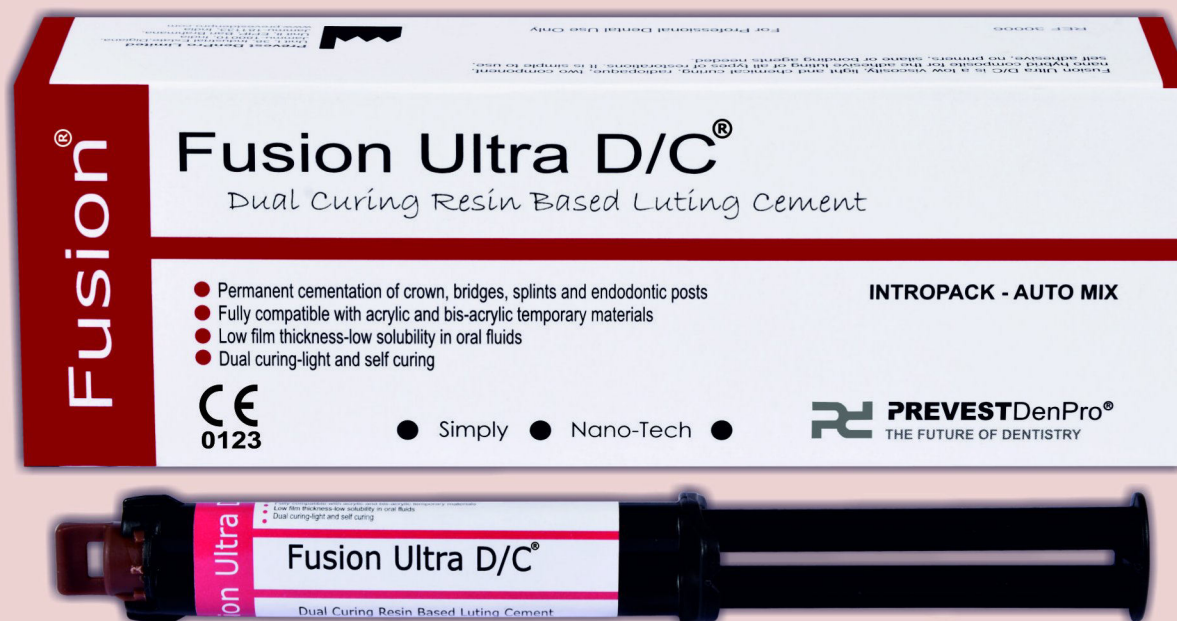

Fusion Ultra D/C® is a low viscosity, light and chemically cured, radiopaque, two component nano hybrid luting cement for adhesive luting of indirect composite, zirconium core, metal and porcelain/ceramic restorations to both dentine and enamel.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customercare@prevestdenpro.com