

GINGIVA SHIELD VLC

Light Curing Gingival Resin Barrier, Blue

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Gingiva Shield VLC is a light curing resin based gingival barrier in syringe applicator. It quickly and easily seals the margins to protect soft tissues during whitening treatment and other procedures.

Composition : Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Fumed Silica, Silanated Barium Glass Powder, Pigments, Curing Agents and Stabilizers.

Contraindications :

Contraindicated for patients with known allergies to polymerizable resins.

Instructions for use :

1. Place self-supporting lip/cheek retractors.
2. Remove lock cap and attach applicator tip provided with the product.
3. Rinse and air dry the teeth.
4. Build a resin barrier along the gingival margin about 1.5-2.0 mm thick. Continue building the barrier apically until approx 6 mm of the tissue is covered and extend the resin beyond the last tooth to be bleached.
5. Light cure the resin by fanning movement over the entire surface for 20-30 sec.
6. After completion of whitening procedure, suction of the whitening gel, rinse teeth with water/air syringe. Lift Gingiva Shield Barrier from the surface using an explorer or instrument.

Storage :

Store at temperature between 10° to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of Manufacturing.

Presentation :

Ref. 90010

Intro Pack : 2 x 1.2ml Syringe, 2 x Applicator tips.

Ref. 90010-1

Trial Pack : 1 x 1.2ml Syringe, 2 x Applicator tips.

Ref. 90011

Economy Pack : 4 x 1.2ml Syringe, 4 x Applicator tips.

Disposal conditions :




To be disposed off based on the prevailing rules of the country in which it is used.


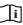
Prevest DenPro Limited
Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India


EC REP
Obelis s.a
Boulevard General Wahlen 53
1030 Brussels, Belgium
Tel: + (32) 2 732 59 54, Fax: + (32) 2 732 60 03
E-mail : mail@obelis.net



CE
0123



Revision :	09
Date :	29-04-2021

LOT Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.