

# HEMOSTAL

## GINGIVAL RETRACTION GEL

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Hemostal Gel is a 25% Aluminum Chloride Gel intended to be used to restrict and stop bleeding and control fluids in the sulcus surrounding the tooth during dental procedure in the mouth

**Composition :** Aluminum Chloride, Glycerine, Fillers, Coloring Pigment, Flavoring agents & Purified Water.

**Intended Purpose / Intended Use :** It is used as a topical hemostatic agent to contact gingival bleeding.

**Indications for Use :** Restrict and stop bleeding during restorative procedures, Crown preparations & oral prophylaxis

**Instructions for use :** Remove the cap from syringe and fix applicator tip. Apply a sufficient amount of gel directly into the sulcus, carefully following the contour of the prepared tooth. Leave in place for 2 minutes. In case bleeding persists, wash off the gel and re-apply and wait for 2 minutes. Rinse gently, dry and take the impression. For Sulcus retraction, the gel impregnated retraction cord should be carefully placed stepwise in the gingival sulcus with the help of a cord. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

### Warnings :

1. Device to be used by a qualified dentist only.
2. Do not inject the material.
3. For topical use in dental operator for control of bleeding only.

### Precautions :

1. Avoid contact with skin and eyes. Do not take internally.
2. Eye contact - If the product comes in contact with eyes, it may cause irritation. Rinse immediately with water and seek medical attention.
3. Skin contact - Avoid contact with skin, it may cause skin sensitization. Wipe off thoroughly with cotton and then wash well with soap and water.

### Contraindications :

1. Do not use the product if the patient is allergic to any of its ingredients.
2. Not to be used for contact of bleeding during surgical extraction of teeth.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

### Presentation :

Ref: 60002

1 x 3g Syringe, 10 x Applicator Tips

### Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

CE

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahma,  
Jammu-181133, India.  
www.prevestdenpro.com  
Customer Care No. : + 91 8899074151  
Email : customercare@prevestdenpro.com  
Made in India

Issue/Revision :	01/00
Date :	24-01-2023

- UDI** Symbol for "UNIQUE DEVICE IDENTIFIER"  
**X** Symbol for "HARMFUL IRRITANT" **☠** Symbol for "CORROSIVE"  
**LOT** Symbol for "BATCH CODE" **🕒** Symbol for "USE-BY DATE" **⚠** Symbol for "NON STERILE" **☂** Symbol for "KEEP DRY"  
**⚠** Symbol for "CAUTION" **📖** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"  
**EC/REP** Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"  
**👶** Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **🏭** Symbol for "MANUFACTURER" **🚫** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  
**🌡** Symbol for "TEMPERATURE LIMIT" **☀** Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.