

# HEMOSTAL

## GINGIVAL RETRACTION SOLUTION

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Hemostal is 25% Buffered Aluminium Chloride solution intended to be used to restrict and stop minor bleeding and control fluids in the sulcus surrounding the tooth during dental procedure in the mouth.

**Composition :** Aluminium Chloride, Glycerine, Purified Water and Excipients.

**Intended Purpose / Intended Use :** It is used as a topical hemostatic agent to control gingival bleeding.

**Indication for Use :** Restrict and stop bleeding during restorative procedures, Crown preparations & oral prophylaxis

**Instructions for Use :** Apply directly to the required place with saturated cotton pellet. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

### Precautions :

1. Avoid contact with skin & eyes. Avoid inhalation of fumes. Do not take internally.
2. Eye contact - If the product comes in contact with eyes, it may cause irritation. Rinse immediately with water and seek medical attention.
3. Skin contact - Avoid contact with skin, it may cause skin sensitization. Wipe off thoroughly with cotton and then wash well with soap and water.

**Contraindications :** Do not use in patients if allergic to any of its components.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf life :** 3 years from the date of manufacturing.

### Presentation :

Ref : 60001

1 x 15ml Glass Bottle, Dispensing Dropper.

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.


CE

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park,  
(EPIP), Bari Brahmna, Jammu-181133,  
India. www.prevestdenpro.com  
Customer Care No. : + 91 889074151  
Email : customercare@prevestdenpro.com  
Made in India

Revision :	10
Date :	19-05-2023

**UDI** Symbol for "UNIQUE DEVICE IDENTIFIER"

 Symbol for "HARMFUL IRRITANT"



 Symbol for "CORROSIVE"

**LOT** Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

**ECI/REP** Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.