

HEMOSTAL

GINGIVAL RETRACTION SOLUTION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Hemostal is 25% Buffered Aluminum Chloride solution intended to be used to restrict and stop minor bleeding and control fluids in the sulcus surrounding the tooth during dental procedure in the mouth

Composition :

Aluminum Chloride, Glycerine, Purified Water and Excipients.

Intended Use :

Restrict and stop bleeding during restorative procedures, Crown preparations & oral prophylaxis

Instructions for Use :

Apply directly to the required place with saturated cotton pellet.

Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref : 60001

1 x 15ml Glass Bottle, Dispensing Dropper.

CE


Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.




Prevest DenPro Limited
Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India

Revision :	07
Date :	24-07-2020

 Symbol for "HARMFUL IRRITANT"

 Symbol for "CORROSIVE"

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.