

HIFLEX IMPRESSION COMPOSITION

THERMOPLASTIC IMPRESSION COMPOUND



Product Description :

Red color Thermoplastic impression material in the form of round plates cakes to take primary impression in edentulous cases.

Indications :

Because of its unique slow setting characteristics Hiflex Impression Composition functional enables production of extremely accurate impressions. This makes it particularly useful as an impression material for rebasing and muscle trimming. May also be used for general impressions such as dentures, root surfaces and posts.

Instructions for Use :

Immerse the required quantity of Hiflex Impression Composition in hot water (55 °C / 132 °F). When softened knead thoroughly with the fingers and place in dry impression tray. Before inserting in the mouth, pour a little cold water over the back of the impression tray. Glaze the surface with dry heat (over a spirit lamp or Bunsen), being careful not to use too much heat, and quickly insert into the mouth. After removal, put in to cold water.

Precautions :

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. In case of contact with eyes, remove and flush with water.
3. In rare cases the products may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

3 years under proper storage.

Presentation :

Ref: 70012

5 round plates / cakes, Net wt : 200g

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited


Unit II, EPIP, Bari Brahma, Jammu-181133, India.


www.prevestdenpro.com


Customer Care No. : 1800-180-7172


Email : customercare@prevestdenpro.com


Revision :	02
Date :	23-05-2019

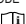
 Symbol for "BATCH CODE"


 Symbol for "USE BY"


 Symbol for "NON STERILE"


 Symbol for "KEEP DRY"


 Symbol for "CAUTION"


 Symbol for "CONSULT INSTRUCTION FOR USE"


 Symbol for "CATALOGUE NUMBER"


 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"

 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"

 Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.