

Product Description :

Hiflex Modelling Wax pink sheets is specially formulated for use in Dentistry technique. Made of pure, natural bees wax and resins, so that it softens easily under hot water. Easy to carve, suitable for a variety of Dental techniques.

Composition :

Beas Wax, Paraffin Wax, Microcrystalline Wax, Mineral oil and Colorant.

Indications :

- ◆ As an articulating medium.
- ◆ For preparing bite block for occlusal registration.
- ◆ For preparing spacer for custom tray.
- ◆ Sheets of pink wax for modelling dentures bases and acrylic saddle.
- ◆ Making wax pattern for complete and partial dentures.
- ◆ Making wax pattern of Orthodontic appliances.

Instruction for Use :

Before use plastify the sheets of wax by dipping it in warm water or holding over the flame of a burner. It is easy to mould when softened and not tear to flakes. Can be used in :

Direct technique : Wax pattern is made directly in oral cavity.

Indirect technique : Wax pattern is made on cast/die in lab outside the oral cavity.

Contraindications :

It is contraindicated for use in patient who have a history of severe allergic reaction to any of the ingredient.

Storage Conditions :

Store in a cool and dry environment between 10°C to 24°C.

Shelf life :

3 years from date of manufacture.

Presentation : Types (Soft, Regular, Hard)

Ref : 13002

12 Sheets in 225gm mono pack.

Ref : 13003

24 sheets 500gm mono pack.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.














www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com



Revision :	02
Date :	23-05-2019

-  Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"
-  SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.