

HIFLEX SHELLAC BASE PLATE

Product Description : Hiflex Shellac Base Plates are with optimum thickness for high mechanical strength. They are easy to manipulate with shape stability after cooling down and with no adhesion to model material or cast. Hiflex Shellac base plates are ready to use Maxillary & Mandibular plates.

Composition : Base Plates contain : Shellac, Stearin, Talcum, Mica.

Colour : Light brown / pink

Indications :

- ◆ Individual custom impression trays.
- ◆ Making denture : Base material for bite impressions and tooth line ups.
- ◆ Modelling works.

Instruction for Use :

Place the base plate on a gypsum model and heat it with laboratory burner flame. Soften the base plate with a fanned flame and press onto the plaster cast with your fingers. When heat through enough, the plate adheres to the gypsum model. Final adaption can be carried out e.g., using a wet wiping cloth. After adaption cut the excessive parts of the plate with a hot modelling knife.

Precautions :

1. To prevent possible damage, never press an insufficiently heated base plate onto a plaster cast.
2. Move the flame round so as to avoid local overheating and deformation of the plate.

Contraindications :

In rare cases, the product may cause irritation to some persons, if such reactions or experienced discontinue the use of product and refer to physician.

Storage Conditions :

Store in a cool and dry environment between 10°C to 24°C.

Shelf life :

3 years from date of manufacture.

Presentation :

Ref: 13001

20 upper / lower base plates in mono box available in thickness : 1.3mm

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,
Jammu-181133, India.










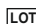


www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customer-care@prevestdenpro.com



Revision :	02
Date :	23-05-2019

-  **LOT** Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"
-  SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.