

Micron® Dentin Conditioner

Polyacrylic Acid Cavity Conditioner

Micron Dentin Conditioner is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine, thus enhancing the bond between glass ionomer cement and the dentine.



Indications

- Removal of dentinal Smear Layer.
- To Condition the dentine.

Benefits

- Polyacrylic Acid solution enhance the bond between glass ionomer cement and the dentine.
- Rapid conditioning with low PH value.

Chemistry

Liquid : 10% Polyacrylic Acid Solution.


Technical Data

- 1) PH : 1.5 – 2
- 2) Colour : Blue


Techniques



Cavity preparation

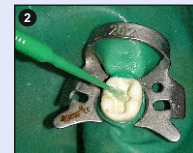

Application of dentin conditioner
To the bonding surfaces for 20 sec

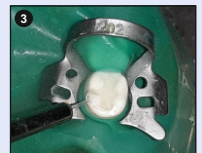

Rinse thoroughly
with water



Dry by blotting with cotton
pellet



Restored cavity with
Micron Superior

Presentation

Ref. : 30010-1
1 x 10 ml Dropper Bottle, 1 x Instruction Manual.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com

Micron® Dentin Conditioner

Polyacrylic Acid Cavity Conditioner



Micron Dentin Conditioner is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine, thus enhancing the bond between glass ionomer cement and the dentine.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com