

**Liquid for temporary cementation with zinc oxide**

## Product Description :

Has an obtundent effect and reacts with zinc oxide to form a temporary cement for sedative dental dressing or lining under non-composite resin restoration.

**Composition :** Eugenol and Vegetable Oils.

## Indications For Use :

For temporary relief of localized dental pain. May be combined with Zinc Oxide to form a temporary sedative dressing.

## Instruction for use :

Place a drop on a cotton bud and gently apply to the inflamed area. Mix with Zinc Oxide to form a thick paste and place into a tooth cavity either as a dressing on a painful tooth or as a lining under a non composite resin restoration.

## Warning :

Contact with the skin may cause irritation, burns or hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components

## Precautions :

Avoid contact with eyes. Inhibits the polymerization of composite resin and should not be used as a dressing prior to placing a composite resin restoration

## Storage Conditions :

Store in a cool and dry environment between 10°C to 24°C. Always close the cap immediately after use. Keep Away from Moisture. Shelf Life not affected after opening if stores in a cool dry place. If not to be used for a longer duration, keep them in refrigerator. Allow material to reach room temperature prior to use.

**Shelf Life :** 3 years from the date of manufacturing.

## Presentation :

Ref: 40038

1x 15ml Glass Bottle, 1 x Dispensing Dropper

Ref: 40039

1x 30ml Glass Bottle, 1 x Dispensing Dropper

Ref. : 40040

1x 110ml Glass Bottle, 1 x Dispensing Dropper

**Disposal Conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

### Prevest DenPro Limited

Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com


Made In India


Revision :	00
Date :	03-03-2020

 **LOT** Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON-STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.