

# MICRON DENTIN CONDITIONER

## POLYACRYLIC ACID CAVITY CONDITIONER

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Micron Dentin Conditioner is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine, thus enhancing the bond between glass ionomer cement and the dentine.

**Composition :** 10% Polyacrylic Acid Solution.

### Indication for Use :

1. After tooth preparation, apply Micron Dentin Conditioner to the bonding surfaces for 20 seconds using a cotton pellet or sponge.
2. Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).
3. Apply Micron Superior or Micron Luting or other glass ionomer as a bonding agent, base/liner or restorative material.
4. Close the bottle immediately after use.

**Contraindications :** In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### Caution :

1. In case of contact with oral tissue or skin, flush immediately with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

**Storage Conditions :** Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf life :** 3 years from date of manufacture.

**Presentation :**

**Ref. 30010-1**

1x10 ml Dropper Bottle, 1xInstruction Manual.


**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : 1800-180-7172  
Email : [customer-care@prevestdenpro.com](mailto:customer-care@prevestdenpro.com)


EC REP  
Obelis s.a  
Boulevard General Wahis 53  
1030 Brussels, Belgium  
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

CE  
0123

Revision :	05
Date :	23-05-2019

**LOT** Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

**EC/REP** SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.