

# ORAFIL-G

## TEMPORARY FILLING MATERIAL

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Orafil-G is a paste consistency, radiopaque, temporary filling material which cures upon contact with oral fluids. It possesses high adhesives properties and therefore offers best possible closure of dental cavities

**Composition :** Zinc oxide, Calcium Sulphate, Zinc Sulphate, Dibutyl Phthalate, EVA Resin, Spearmint oil.

**Indications :** Recommended for temporary restoration of cavities in the interval between tooth preparation and the permanent cementation of completed inlays.

**Instructions for Use :** Use any suitable instrument to introduce the paste directly into wet cavity. Press and spread with the instrument to fill the cavity and adapt to the cavity walls. Remove any excess paste by wiping it off with a moist cotton pellet towards the margins of the cavity. Hardened paste can be removed with the use of rotary instrument or any suitable hand instrument. It is advised not to fill very deep cavities up to the base of cavity.

**Warning :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis) and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Incompatibility :** In sensitive persons, the product may cause sensitization. If allergic reaction occurs remove the filling immediately and consult physician.

**Contraindications :** Do not apply directly on exposed pulp.

**Storage Conditions :** Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

**Presentation :**

**REF : 11001**

**Presentation :** 1x 40g Jar.

**Disposal Conditions :**

To be disposed off based on the Prevailing rules of the country in which it is used.

### Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,  
Jammu-181133, India.

[www.prevestdenpro.com](http://www.prevestdenpro.com)

Customer Care No. : 1800-180-7172





Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)




Made In India







**CE**  
0123



Revision :	06
Date :	24-07-2020

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.