

# ORAFIL-LC

## Light Cured Temporary Filling Material



**Product Description :** Orafil-LC is a light-curing material for temporary filling and sealing. The elastic consistency assures an easy removal, even in case of undercuts, and finishing the cavity is not necessary. Thus the fitting of the temporary filling will be improved. Orafil-LC is available in syringes for direct application into the cavity.

**Composition :** Urethane Dimethylacrylate, Primer, Fillers, Photoinitiators, Stabilizers & Pigments.

**Intended Purpose / Intended Use :** It is intended to be used as a temporary filling materials.

**Indications for Use :** All types of temporary fillings, temporary protection in inlay and onlay technique.

**Instructions for Use :**

**Application :**

1. Preparation, lining and impression to be carried out as usual. Rinse and dry the cavity.

2. Apply Orafil-LC into the cavity with a suitable instrument (round condenser, spatula, etc.)

**Handling :** Contour with slight excess. For use in the inlay and onlay technique, keep the approximate contact points free of resin with dental floss or a matrix before curing to maintain tooth mobility. Cure for 20 - 40 s (depending on layer thickness) with a conventional polymerisation device. The light output should be a minimum of 500 mW/cm<sup>2</sup> on halogen polymerisation devices and 300 mW/cm<sup>2</sup> on LED devices. As a rule, 4 mm deep layers are sufficiently cured in 40 s. Increased exposure times are required when the light source cannot be brought close enough to the material. Check occlusion. Excess can be removed with a scalpel. Silicone points are suitable for grinding.

**Removal of the temporary filling :** Insert a rigid instrument into the temporary filling and extract it. Simply stick in a probe and remove the filling. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :**

1. Avoid use of lining material containing eugenol as it may alter setting time of orafil LC.

2. Adhesion of dental resins or lining (eg. light curing materials) may occur. Thus, apply suitable insulating agent (eg. glycerine, vaseline).

**Contraindications :** Do not use in patients, If allergic to methacrylate resins.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 Years from the date of manufacturing.

**Presentation :**

Ref: 11002-1

1 x 3g Syringe

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
 Unit II, Export Promotion Industrial Park,  
 (EPIP), Bari Brahmana, Jammu-181133,  
 India. www.prevestdenpro.com  
**Customer Care No. :** + 91 8899074151  
**Email :** customercare@prevestdenpro.com  
 Made in India

**EC REP**  
 Obelis s.a  
 Boulevard General Wahnis 53  
 1030 Brussels, Belgium  
 Tel: +(32) 2 732 59 54. Fax: +(32) 2 732 60 03  
 E-mail : mail@obelis.net

**CE**  
 0123  

Revision :	05
Date :	02-09-2023

**UDI** Symbol for "UNIQUE DEVICE IDENTIFIER"

**LOT** Symbol for "BATCH CODE"   Symbol for "USE-BY DATE"   Symbol for "NON STERILE"   Symbol for "KEEP DRY"  
 Symbol for "CAUTION"   Symbol for "CONSULT INSTRUCTION FOR USE"   **REF** Symbol for "CATALOGUE NUMBER"  
**EC REP** Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"  
 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"   Symbol for "KEEP OUT OF THE REACH OF CHILDREN"   Symbol for "MANUFACTURER"  
 Symbol for "TEMPERATURE LIMIT"   Symbol for "KEEP AWAY FROM SUNLIGHT"   **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.