

PF Seal®

Light Cured Pit & Fissure Sealant

PF SEAL is a light cured sealant which is recommended for prevention of caries by sealing pit and fissure. It contains Amorphous Calcium Phosphate (ACP) Having Excellent Remineralizing Properties.



Indications

- Sealing / Filling of the pit and fissure.
- Filling of the small caries.
- Sealing / facing of damaged surfaces.
- Covering of caries predilection sites during orthodontic treatment.

Benefits

- New Resin Formula For Strong & Long Lasting Sealant Contains Amorphous Calcium Phosphate (ACP).
- Convenient Flow Through Tips.
- Unique combination of fissure sealing with active remineralization.

Chemistry

Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Silanated Barium Glass Powder, Amorphous Fumed Silica, Curing Agents and Stabilizer.


Technical Data

- 1) Curing Time : 40 Sec.
- 2) Depth of Cure : 1.5 – 2mm


Techniques



Pit & Fissure Caries

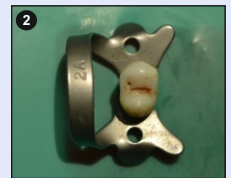


Cavity preparation

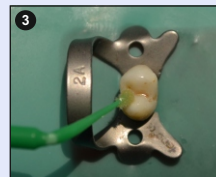

Application of bonding agent



Placement of PF Seal

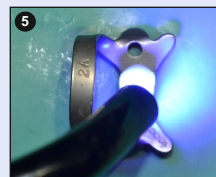

Light cure for 40 sec



Completed restoration

Presentation

Ref. : 20012-1

Trial Pack : 1 x 2g Syringe, 5 x Applicator Tips.

Ref. : 20012

Intro Pack : 2 x 2g Syringe, 10 x Applicator Tips.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548

Tel.: 91 1923 222774/222779

Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com

Website : www.prevestdenpro.com

E-mail : customer@prevestdenpro.com

PF Seal®

Light Cured Pit & Fissure Sealant



PF SEAL is a light cured sealant which is recommended for prevention of caries by sealing pit and fissure. It contains Amorphous Calcium Phosphate (Acp) Having Excellent Remineralizing Properties.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customercare@prevestdenpro.com