

Platina Hi-Gloss®

Composite Polishing Paste

Platina Hi-Gloss is an aluminium oxide paste for final polishing of direct or indirect anterior and posterior resin restorative materials. It contains potassium nitrate topical desensitizer for patient comfort.



Indications

- Final polishing of direct and indirect anterior & posterior composite restorations / veneers.

Benefits

- Non splatter-non drying formula Spearmint flavored With potassium nitrate topical desensitizer.
- High luster polished surface.
- Water soluble Aluminium Oxide paste.


Chemistry

Glycerin, White Fused Alumina, Polyethylene Glycol, Amorphous Silica And Spearmint Oil.


Technical Data

- 1) Consistency : 29 mm
- 2) Film Thickness : 60 µm


Techniques

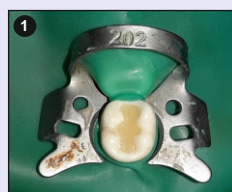

Composite Restoration

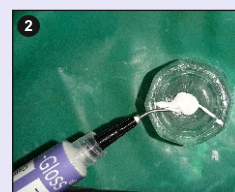

Dispense a portion of Platina Hi-Gloss polishing paste onto a Dappen Dish

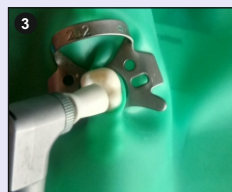

Apply Small Amount of Platina Hi-Gloss polishing paste onto a enhance disc and then rinse from tooth surface and enhance cup.



Smooth & polished surface after polishing

Presentation

Ref. : 50001
1 x 4g Syringe, 1 x Instruction Manual.

PREVEST DENPRO LIMITED


Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com

Platina Hi-Gloss®

Composite Polishing Paste

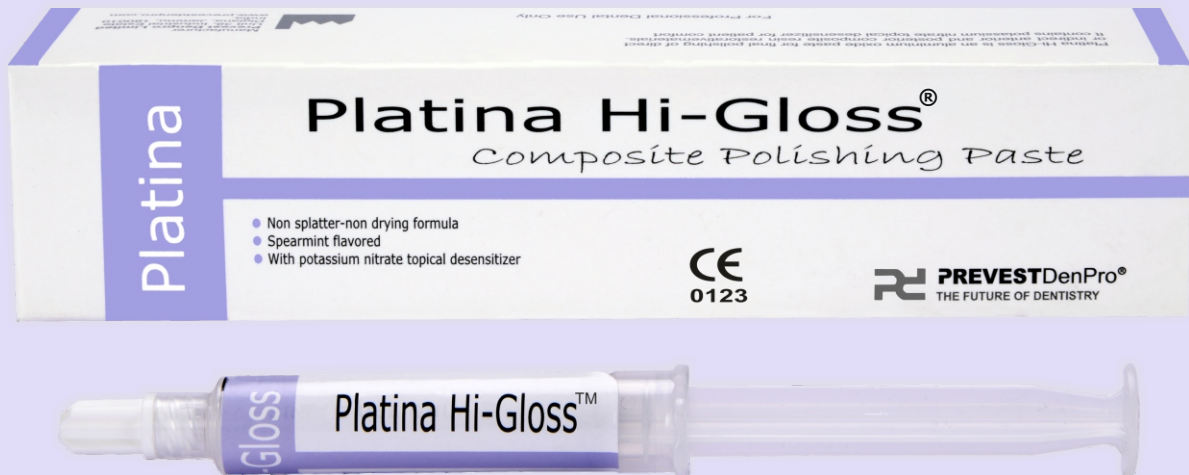

Platina Hi-Gloss is an aluminium oxide paste for final polishing of direct or indirect anterior and posterior resin restorative materials. It contains potassium nitrate topical desensitizer for patient comfort.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customer@prevestdenpro.com