

PULP-X

PULP DEVITALIZER

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Pulp-X is a devitalizer paste which provides powerful pulp devitalizing properties due to paraformaldehyde. It is a fast acting and pain relieving paste, easy to apply.

Composition :

Formaldehyde, Fillers, Parachlorophenol, Pigments and Excipients.

Intended purpose/Intended use : It is intended to be used as pulp devitalizer.

Indications for use :

Pulp Devitalizer prior to mortal extirpation or amputation. Residual devitalization after removal of non-vital pulp tissue.

Instructions for use :

Eliminate as far as possible, the dentine of the decayed cavity in order to ensure close contact with the pulp. Close contact of the paste with the pulp accelerates the devitalizing process. Insert a pellet of paste into the base of the cavity and seal hermetically with a temporary cement. Devitalization is completed within five to seven days. After devitalization, a permanent root canal filling can be placed once the pulp is totally removed and pain relief is complete.

The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Warnings :

Contains formaldehyde which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions :

1. The cavity must be carefully sealed with the cement to prevent any leaching of the product which can cause local destruction of the periodontium.

2. The cap should always be replaced immediately after use.

Contraindication : Do not use in patients if allergic to formaldehyde or any other ingredients.

Storage Conditions :

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

3 years from the date of manufacturing.

Presentation :

Ref : 40022

1 x 3g Syringe, 10 x Applicator Tips

Ref: 40022-1

2 x 3g Syringe, 10 x Applicator Tips

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.





 **Prevest DenPro Limited**
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India





Obellis s.a
Boulevard General Wabis 53
1030 Brussels, Belgium
Tel: +132 2 732 99 54, Fax: +132 2 732 60 03
E-mail : mail@obellis.net



0123




Revision :	10
Date :	24-01-2023




 Symbol for "UNIQUE DEVICE IDENTIFIER"

 Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.