

# Renew Universal™

One Component Self Etch Universal Bonding Agents

Renew universal is a light cured single component dental bonding agents containing-etching, priming and bonding in one bottle and contains unique combination such as 10-MDP and 4-META. Compatible with light cured and dual cured composite materials.



## Indications

- Direct bonding of light-cured composites, compomers and ormocers to tooth structure.
- Bonding of dual-cured core build up composites to tooth structure as long as these materials are light-cured.
- Intraoral repair of composite, metal-based and zirconia/alumina-based restorations.
- Treatment of hypersensitive teeth.

## Benefits

- Excellent bonding capabilities to dentin and cut /under cut enamel.
- Fully compatible with light cured and dual cured composite materials.

## Chemistry

A novel dental adhesive containing a unique combination of functional monomers such as 10-MDP And 4- META.


## Technical Data

- 1) Curing Time : 20-30 sec.
- 2) Shear Bond Strength : 20-30 MPa


## Techniques




Prepared cavity

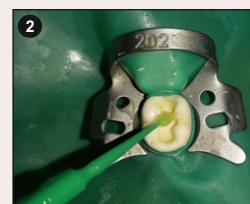

Apply & leave undisturbed for 10 sec after the application

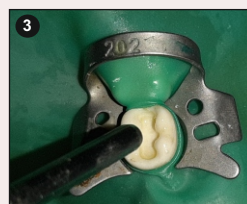

Dry thoroughly for 5 sec with oil free air

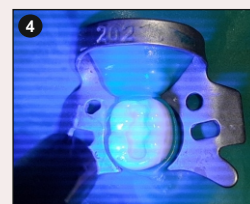

Light cure for 20 sec

## Presentation

Ref. : 10017  
Intro Pack : 1 x 1.2ml Syringe, 5 x Applicator Tips, 1 x Instruction manual.  
Ref. : 10018  
Regular Pack : 1 x 5ml Bottle, 1 x Instruction manual.  
Ref. : 10019  
Kit Pack : 2 x 5ml Bottle, 40 x Micro Brushes, 1 x Instruction manual.

## PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),  
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548  
Tel.: 91 1923 222774/222779  
Toll Free No. : 1800-180-7172

E-mail : [info@prevestdenpro.com](mailto:info@prevestdenpro.com)  
Website : [www.prevestdenpro.com](http://www.prevestdenpro.com)  
E-mail : [customer@prevestdenpro.com](mailto:customer@prevestdenpro.com)

# Renew Universal™

One Component Self Etch Universal Bonding Agents



Renew universal is a light cured single component dental bonding agents containing-etching, priming and bonding in one bottle and contains unique combination such as 10-MDP and 4-META. Compatible with light cured and dual cured composite materials.

## PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),  
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548  
Tel.: 91 1923 222774/222779  
Toll Free No. : 1800-180-7172

E-mail : [info@prevestdenpro.com](mailto:info@prevestdenpro.com)  
Website : [www.prevestdenpro.com](http://www.prevestdenpro.com)  
E-mail : [customer-care@prevestdenpro.com](mailto:customer-care@prevestdenpro.com)