

# REVEAL

## CARIES INDICATOR

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Reveal is a green FD&C dye in a propylene glycol base. Reveal assists the clinician in visual differentiation between the infected carious dentine and sound and re-mineralizable dentine by its ability to selectively dye the carious dentine.

**Composition :** Green F D & C dye, Propylene Glycol.

### Indications :

Caries indication by visual differentiation between infected carious dentine & sound, remineralizable dentine.

### Instructions for Use :

Make conservative cavity access. Wash and dry carious dentine. Dispense 1 or 2 drops of Reveal in to a dappen dish and apply to the cavity with a brush or cotton pellet. Wait for 5 to 10 seconds and rinse with water spray. Remove green stained infected dentine. Repeat the application until carious dentine is removed and no further dentine staining occurs.

### Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

### Caution :

A low speed rotary instrument should be used for removing the carious dentine since frictional heat generated by the high speed tool can cause pain before all the stained outer carious dentine is removed.

### Contraindications :

Reveal does not contain any hazardous ingredients in sufficient concentration to be harmful to human beings and causing irritation to oral tissues when not used as directed.

### Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

### Shelf Life :

3 Years from the date of manufacturing

### Presentation :

Ref: 80002

1 x 5ml plastic dropper bottle.

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

### Prevest DenPro Limited

Unit II, EPIP, Bari Brahmana,

Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : 1800-180-7172

Email : customercare@prevestdenpro.com

Made In India

EC REP

Obelis s.a

Boulevard General Wahijs 53

1030 Brussels, Belgium





Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03




E-mail : mail@obelis.net

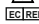

CE  
0123



Revision : 06



Date : 24-07-2020

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.