

IFU Size : 161 H x 104 W

## SHIELD ACTIV DENTIN DESENSITIZER

**PREVEST**DenPro®  
THE FUTURE OF DENTISTRY

**Product Description :** Shield Activ Dentin Desensitizer is a highly effective, one step, chair-side treatment for dentinal hypersensitivity caused due to cervical erosions and exposed roots. It acts very fast to seal dentinal tubules. No surface layer is formed and is therefore ideal for use under all indirect restorations. It contains 0.5% NaF equivalent to 0.23% of Fluoride (2300ppm).

**Composition :** Solvent, Water, Sodium Fluoride, Potassium Nitrate, Primer.

**Intended Purpose / Intended Use :** It is intended to be used for the treatment of hypersensitive teeth.

**Indications for Use :** Shield Activ Dentin Desensitizer is used for treatment of dentinal hypersensitivity, before and after scaling, under veneers, inlays crowns, onlays, amalgams, composites, before and after tooth bleaching or whitening.

**Instructions for Use :** Clean the dentine and rinse with water. Dry the surface to be treated. Apply smallest amount of shield Activ Desensitizer to the dentinal surface in thin layer with a pellet or brush and leave for 30-60 seconds. Dry the surface by applying a stream of compressed air. Repeat the applications two or three times. In case of persisting symptoms, reapply at one week interval. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

**Warnings :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :** Shield Activ Desensitizer should be used by a dentist. Do not allow the product to come in contact with eyes or skin. Do not breathe vapor. Advise patient to avoid brushing of treated area and consumption of acidic food or drinks for 24 hours.

### Contraindications :

1. Do not use the product for patients with hypersensitivity to the product's components.
2. Do not use in case of pulp inflammation in dentinal area.

**Storage Conditions :** Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf life :** 3 years from the date of manufacturing.

### Presentation :

Ref : 80003

1 x 5ml Squeeze Bottle.

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park (EPIP),  
Bari Brahmmana, Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : + 91 8899074151  
Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made in India

EC REP  
Obellis s.a  
Boulevard General Wahnis 53  
1030 Brussels, Belgium  
Tel: +32(0) 2 725 69 54, Fax: +32(0) 2 725 69 63  
E-mail : [mail@obellis.net](mailto:mail@obellis.net)

CE  
0123  
Issue/Revision : 01/01  
Date : 19-02-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"    USE-BY DATE Symbol for "USE-BY DATE"    NON STERILE Symbol for "NON STERILE"    KEEP DRY Symbol for "KEEP DRY"  
CAUTION Symbol for "CAUTION"    CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE"    REF Symbol for "CATALOGUE NUMBER"  
EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"  
DO NOT USE IF PACKAGE IS DAMAGE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"    KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN"    MANUFACTURER Symbol for "MANUFACTURER"  
TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT"    KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT"    MD Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.