

# SPECTRA

## PROPHYLAXIS PASTE

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Spectra Prophylaxis Paste is a cleaning and polishing paste with a unique combination of abrasive materials zirconium silicate and pumice powder for cleaning and polishing during dental procedures. It contains sodium fluoride at a concentration of 1% fluoride ion for post treatment comfort to the patient. Spectra Prophylaxis paste is produced in a variety of fruit flavors for pleasant feeling during the treatment procedure.

**Composition :** Zirconium Silicate, Pumice Powder, Glycerine, Carboxy Methyl Cellulose, Fumed Silica, Sodium Fluoride, Pigments and flavoring matter.

**Indications :** Prophylaxis Paste, is indicated for use to effectively clean and polish tooth structure with minimal abrasion

**Contraindications :** Prophylaxis Paste is contraindicated in patients with a known allergy to any of the components.

### Instructions for use :

1. Dispense Prophylaxis Paste into dappen dish or other suitable container. A saliva ejector may be helpful during the actual prophylaxis.

2. Fill rubber prophylaxis cup with paste. Before starting hand piece, carry paste to field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste be provided for each surface in order to obtain maximum cleaning effectiveness.

3. Run rubber cup at SLOW SPEED over each tooth surface for approximately 10 seconds. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.

4. A rubber cup or soft bristle brush may be used on occlusal surfaces.

**Warning :** Prophylaxis Paste may cause eye irritation upon contact. If contact with eyes occurs, immediately flush eyes with copious amount of water for 15 minutes. Seek medical attention if irritation or discomfort persists.

### Precautions :

1. All tooth surfaces should be thoroughly scaled to remove calculus.

2. Necessary examination and radiographs should be taken prior to prophylaxis with Prophylaxis Paste.

3. During use, wear protective glasses and gloves.

**Storage Conditions :** Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of Manufacturing.

### Presentation :

340g Plastic Jar, Fresh Mint (Ref : 50005-1) / 200g Plastic Jar, Fresh Mint (Ref : 50005) / 75g Plastic Jar, Fresh Mint (Ref : 50006) / Fresh Fruit (Ref : 50007) / Fresh Strawberry (Ref : 50008) / Fresh Orange (Ref : 50009) / Fresh Pineapple (Ref : 50010).

**Disposal conditions :** To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmana,  
Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : 1800-180-7172  
Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made In India

EC REP  
Obelis s.a  
Boulevard General Wahnis 53  
1030 Brussels, Belgium  
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 63  
E-mail : [mail@obelis.net](mailto:mail@obelis.net)

CE  
0123

Revision :	09
Date :	24-07-2020

**LOT** Symbol for "BATCH CODE" **USE BY** Symbol for "USE BY" **NON STERILE** Symbol for "NON STERILE" **KEEP DRY** Symbol for "KEEP DRY"

**CAUTION** Symbol for "CAUTION" **CONSULT INSTRUCTION FOR USE** Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

**EC REP** SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE **DO NOT USE IF PACKAGE IS DAMAGE** Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

**KEEP OUT OF THE REACH OF CHILDREN** Symbol for "KEEP OUT OF THE REACH OF CHILDREN" **MANUFACTURER NAME & ADDRESS** Symbol for "MANUFACTURER NAME & ADDRESS"

**TEMPERATURE LIMITATION** Symbol for "TEMPERATURE LIMITATION" **KEEP AWAY FROM SUNLIGHT** Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.