

# UltraDry

## Drying Agent

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** UltraDry is a fast drying, cleaning and degreasing agent for dental surfaces.

### Indications :

- ◆ To remove moisture from dental surfaces.
- ◆ To dry tooth surface prior to applying pit and fissure sealants (PF Seal, Prevest DenPro Ltd.) and other materials which are adversely affected by the presence of moisture.

Composition : Ethyl alcohol and an organic solvent.

### Directions for Use :

1. Remove syringe cap and place a red dropper tip on the syringe.
2. Isolate the operative area in the mouth to maintain a clean dry field (for intra oral applications only).
3. Rinse the operative area and air dry with clean, uncontaminated, oil free and moisture free compressed air.
4. Dispense a small amount of UltraDry on the desired surface. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between forefinger and thumb while resting the plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dropper tip for each patient.
5. To promote rapid drying, use a gentle flow of clean, uncontaminated air if desired.

### Warning :

1. UltraDry composition is highly flammable. Handle with care and keep away from open flames.
2. Do not place UltraDry directly on dentin as it can desiccate the dentinal tubules.

### Contra-Indications :

This product is contra-indicated for use in or by individuals who have shown hypersensitivity to any of its components.

### Storage & Shelf Life :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

### Shelf Life :

2-years from the date of manufacturing.

### Presentation :

Ref: 40033

**Intro Pack :** 1 x 1.2ml Syringes + 5 Applicator Tips.

Ref: 40034

**Economy Pack :** 1 x 10ml in a Plastic Bottle.

### Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, EPIP, Bari Brahmna,  
Jammu-181133, India.  
www.prevestdenpro.com  
**Customer Care No. :** 1800-180-7172  
**Email :** customercare@prevestdenpro.com  
Made In India

EC REP  
Obelis s.a  
Boulevard General Wahiis 53  
1030 Brussels, Belgium  
Tel: +(32) 2 732 99 54, Fax: +(32) 2 732 60 03  
E-mail : mail@obelis.net

CE  
0123

Revision :	05
Date :	24-07-2020

- LOT** Symbol for "BATCH CODE"   Symbol for "USE BY"   Symbol for "NON STERILE"   Symbol for "KEEP DRY"
- Symbol for "CAUTION"   Symbol for "CONSULT INSTRUCTION FOR USE"   **REF** Symbol for "CATALOGUE NUMBER"
- EC REP** SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE   Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN"   Symbol for "MANUFACTURER NAME & ADDRESS"
- Symbol for "TEMPERATURE LIMITATION"   Symbol for "KEEP AWAY FROM SUNLIGHT"   Symbol for "FLAMMABLE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.