

Zinc F+®

Zinc Phosphate Cement

Zinc F + is a zinc phosphate cement suitable for the cementation of crowns, bridges, inlays facings and orthodontic brackets. May also be used for cavity lining under all restorative materials, temporary fillings and fillings in deciduous teeth.



Indications

- High strength base
- Cementation of crowns, bridges and inlays
- Temporary filling
- Filling of deciduous tooth.

Benefits

- High strength bases offering good pulpal protection
- Modulus of elasticity close to tooth structure
- Excellent luting cement with high retentive rates

Chemistry

- Methacrylate Monomer, Tricalcium Phosphate,
- Nano Hydroxyapatite, Barium Zirconate, Stabilizers, and Photoinitiators


Technical Data

- Mixing time : 60 -90 seconds, Setting time : 3:15 sec (luting) ; 3:10 sec (Base)
- Film thickness : 22µm, Compressive strength : 105 Mpa


Base Consistency



Dispensing for base/putty consistency
Use one scoop of powder and 2-3 drops of liquid on a clean glass slab

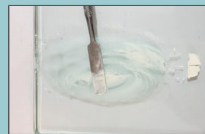

Incorporate the powder into the liquid component in 6-8 increments on a large surface area

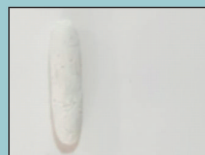

Base/ putty consistency

Luting Consistency



Dispensing for base/putty consistency
Use one scoop of powder and 3-4 drops of liquid on a clean glass slab



Incorporate the powder into the liquid component in 6-8 increments on a large surface area



Luting consistency

Presentation

Ref : 30014

Intro Pack: 30g Powder, 15ml Liquid, 1 x Spoon, 1 x Instruction Manual.

Ref : 30014-1

Economy Pack: 120g Powder, 90ml Liquid, 1 x Spoon, 1 x Instruction Manual

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548

Tel.: 91 1923 222774/222779

Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com

Website : www.prevestdenpro.com

E-mail : customercare@prevestdenpro.com

Zinc F+®

Zinc Phosphate Cement



Zinc F + is a zinc phosphate cement suitable for the cementation of crowns, bridges, inlays facings and orthodontic brackets. May also be used for cavity lining under all restorative materials, temporary fillings and fillings in deciduous teeth.

PREVEST DENPRO LIMITED

Export Promotion Industrial Park (EPIP),
Bari Brahmna, Jammu-181133, India.

Tel.: 91 191 2430442/ 2432548
Tel.: 91 1923 222774/222779
Toll Free No. : 1800-180-7172

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com
E-mail : customercare@prevestdenpro.com