

ZICAL

ROOT CANAL SEALER

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Zical Root Canal Sealer is a resin based sealer in paste-paste form for permanent filling dental root canals. This sealant has been specially designed to tightly seal the apex of the root and the prepared root canal. It can be easily removed along with Gutta Percha Point when refilling is necessary. Zical is radiopaque, non staining, non irritating with excellent thermal insulation. Its long working time and lubricating qualities are additional features. Low zinc oxide contents of the composition alleviate some of the cytotoxic and inflammatory effects of the tissues.

Composition : Zical is a paste-paste composition.

Paste A contains : Zinc oxide, Magnesium oxide, Calcium Phosphate, Bismuth Sub-carbonate, Barium Sulphate, Sodium Borate and Olive oil.

Paste B contains : Vinyl Polysiloxane, Hydrogenated Resin and Fumed Silica.

Instructions for Use : The root canal should be thoroughly clean and dry before filling. Extrude small amount of pastes on mixing pad and mix thoroughly with spatula for 30 seconds to obtain a homogeneous paste. Introduce the paste into the prepared canal with root explorer. Wet the G.P Point with the paste and insert the wet G.P. Point into the Canal. Remove excess GP Point and seal the coronal end with base sealer. Net setting time is 45-60 minutes

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions : Do not overfill the root canals with the paste. Do not use on patients who have history of allergic reactions to any of the components. Avoid contact with eyes.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3years from the date of manufacturing.

Presentation :

Ref: 40010

Auto Mix Pack : 1 x 15g Cartridge, Base & Catalyst Paste, 20 x Automix Tips

Ref: 40011

Manual Mix Pack : 1 x 15g Cartridge, Base & Catalyst Paste, 1 x Mixing Spatula and 1 x Mixing Pad.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmaana,
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : 1800-180-7172


Email : customercare@prevestdenpro.com

Made In India



CE
0123

Revision :	07
Date :	24-07-2020

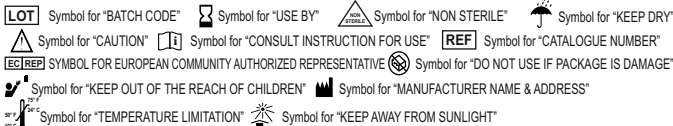

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.