

ZICAL *Ultra*

ROOT CANAL SEALER (POWDER/LIQUID)

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Caution: US federal law restricts this product for sale by or on order of a dentist or physician.

Product Description : Zical Ultra Root Canal Sealer is a zinc oxide-eugenol resin based sealer in powder-liquid form for permanent filling of root canals.

Composition :

Powder: Zinc oxide, Iodoform, Hydrogenated Rosin, Barium Sulphate, Zinc Acetate, Bismuth subcarbonate

Liquid : Eugenol, Olive Oil

Intended use: It is used for permanent filling of dental root canals.

Indications for use: ZICAL ULTRA is used for permanent obturation of the root canal space with the aid of obturating points.

Instructions for use : The root canal should be thoroughly dry before filling. Over filling root canals should be avoided. Both glass slabs and spatula should be clean and thoroughly dry prior to use for mixing cement. Place 1 scoop of powder and 4-5 drops of liquid on the glass slab. Spatulate small portions of the powder into liquid until entire powder is homogeneously mixed. Gradually add more portions of powder to the mix spatulating thoroughly for 2:00-2:30 (min :sec) until a very smooth creamy paste is obtained. Working time is 30 minutes. Introduce the paste into the prepared root canal with a suitable instrument. Insert the Gutta Percha point and seal the coronal end. Net setting time is 45-60 minutes from start of the mix. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Warnings: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation and hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions :

Do not use Zical Ultra with patients that have a history of allergic reaction to Iodine compounds. Avoid contact with eyes

Contraindications: It should not be used with patients who have a history of severe allergic reaction to any of the components.

Storage Conditions :

Store at temperature between 10°C(50 °F) to 24°C(75 °F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life :

3 years from the date of manufacturing.

Presentation :

Ref : 40044

Powder – Liquid Pack : 1 x 15g Powder, 1 x 10ml Liquid, 1 x Spoon, 1 x Mixing Pad, 1 x Dropper, 1 x Instruction Manual

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : +91 8899074151

Email : customercare@prevestdenpro.com

Made in India



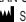
Revision :	00
Date :	02-02-2023




 Symbol for "UNIQUE DEVICE IDENTIFIER"  Symbol for "PRESCRIPTION ONLY"

 Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints & reporting of unexpected events contact customer care.