

ZINC OXIDE

Product Description :

Zinc Oxide Powder reacts with Eugenol to form a sedative dental cement or lining under a non composite resin restoration.

Composition :

Pure Zinc Oxide

Indications :

Zinc Oxide is used for temporary cementation in combination with eugenol.

Instructions for Use :

Mix with Eugenol to form a thick paste and place into a tooth cavity either as a dressing on a painful tooth or as a lining under a non composite resin restoration.

Warning :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions :

Avoid contact with eye. Inhibits the polymerization of composite resins when used with Eugenol and should not be used as a dressing prior to composite resin restoration.

Storage :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref : 40028

Presentation : 1 x 110g Plastic Jar.

Disposal Conditions :




To be disposed off based on the prevailing rules of the country in which it is used.


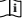
Prevest DenPro Limited
Unit II, EPIP, Bari Brahmna,
Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : 1800-180-7172
Email : customercare@prevestdenpro.com
Made In India


EC REP
Obelis s.a
Boulevard General Wahlen 53
1030 Brussels, Belgium
Tel: +132 2 732 59 94, Fax: +132 2 732 60 63
E-mail : mail@obelis.net



CE
0123



Revision :	06
Date :	24-07-2020

LOT Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC REP SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.